




**EZ FACE SHIELD
CM-5001-EZ**

**INSTRUCTIONS FOR USE
READ INSTRUCTIONS BEFORE USE**

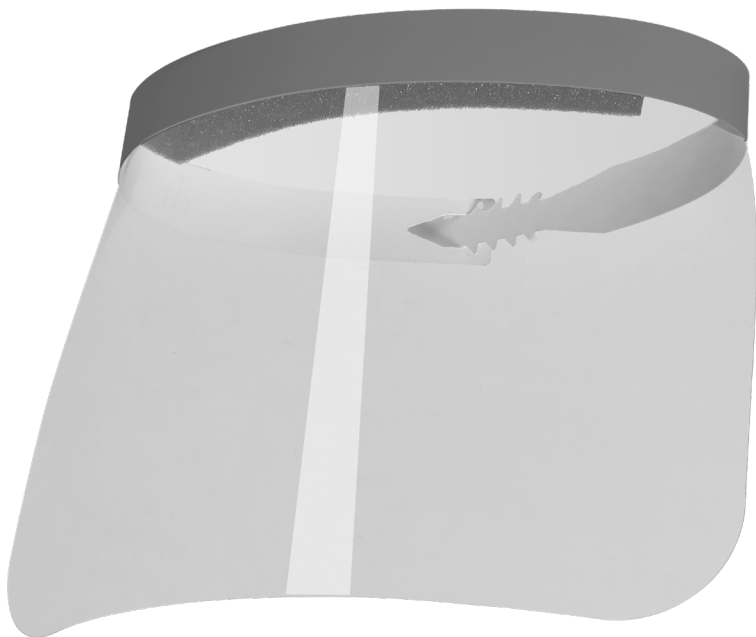

PRODUCT DESCRIPTION

Face shields are used for protection of the facial area and associated mucous membranes (eyes, nose, mouth) from splashes, sprays, and spatter of body fluids. Face shields provide a barrier to acutely-expelled aerosols of body fluids and protect a large area of the face.

INTENDED USE

The Summit Medical Face Shield is intended to provide top, side, and front face protection from aerosols and to minimize airborne cross-contamination.

WARNINGS AND PRECAUTIONS

- Device requires assembly
- **DO NOT RE-USE OR RE-STERILIZE THIS DEVICE.** It is designed for A SINGLE-USE
- This device is supplied clean but is NON-STERILE
- Care should be taken when putting on and taking off the device to prevent contamination
- Care should be taken if it is necessary to adjust the shield during use, as this may lead to contamination
- Ensure that the shield conforms to the face/head and that the shield does not slip while in use
- After use, the shield should be carefully removed without contaminating the eyes, mucous membranes, or clothing with potentially infectious materials and discarded
- Dispose of the device after use to prevent contamination
- Dispose of this device after use in accordance with standard hospital protocol

INSTRUCTIONS FOR USE

Put on the shield, taking care to ensure that it is secure and correctly fitted. To remove, handle via the head band. The shield should be carefully removed without contaminating the eyes, mucous membranes, or clothing with potentially infectious materials and discarded.


CLEANING

This device is non-sterile and is not intended to be disinfected or sterilized.


INCIDENT REPORTING

Any serious incident that has occurred in relation to this device should be reported to the manufacturer.


EZ FACE SHIELD ASSEMBLY INSTRUCTIONS

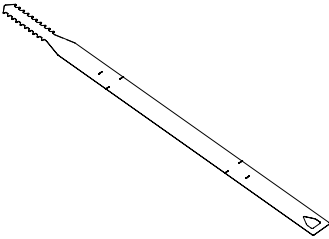


1. Place adjustable band face down, white side up.

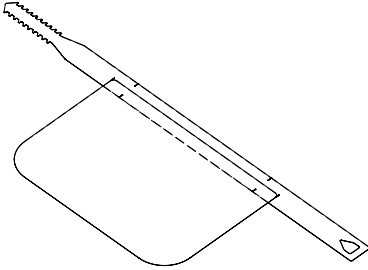

2. Place the shield between the seams only covering half the height of the adjustable band.

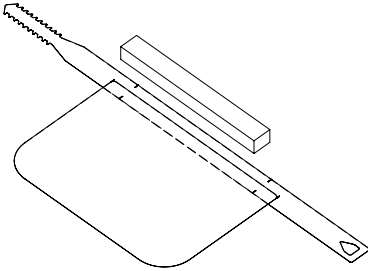



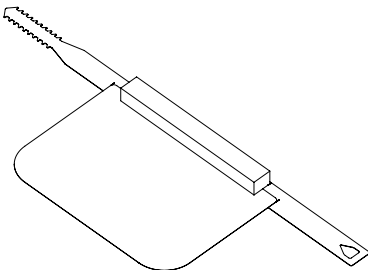






3. Peel backing and adhere foam to both the visor and the adjustable band between the seams.



4. Adjust band length using the self locking arrow latch. Readjust to fit if needed.

Symbol Reference Key

	Lot number/Batch code
	Consult instructions for use
	Catalog number
	Non-sterile
	Manufactured by
	Date of manufacture
	Caution
	Do not use if package is damaged
	Do not reuse



Manufactured by:



Summit Medical LLC
 815 Vikings Parkway, Suite 100
 St. Paul, MN 55121 | USA

P: 1-888-229-2875 | +1 651-789-3939
 F: 1-888-229-1941 | +1 651-789-3979
www.summitmedicalusa.com

This face shield is being offered to the community during this public health emergency. This product is not required to be cleared for medical use by the U.S. Food and Drug Administration. This product has not been evaluated for the prevention of specific diseases or infections, the filtering of surgical smoke or fumes, the filtering of specific amounts of viruses or bacteria, or the killing or reduction of the amount of viruses, bacteria, or fungi. The end user of this device is responsible for decisions regarding appropriate personal protective equipment for their personnel. SUMMIT MEDICAL, LLC. MAKES NO WARRANTY WHATSOEVER WITH RESPECT TO THE FACE SHIELD, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE, WHETHER EXPRESS OR IMPLIED BY LAW, USAGE OF TRADE OR OTHERWISE.