SUMMIT MEDICAL FACE SHIELD CM-5001

INSTRUCTIONS FOR USE / MODE D'EMPLOI /
GEBRAUCHSANWEISUNG / INSTRUCCIONES DE USO /
INSTRUÇÕES DE USO / ISTRUZIONI PER L'USO /
使用説明書

**READ INSTRUCTIONS BEFORE USE / LIRE LES PRÉSENTES INSTRUCTIONS
AVANT UTILISATION / BITTE VOR VERWENDUNG DIE ANWEISUNGEN LESEN /
LEA LAS INSTRUCCIONES ANTES DE USAR / LEIA AS INSTRUÇÕES ANTES DE
USAR / PRIMA DELL'USO, LEGGERE LE ISTRUZIONI / ご使用の前にお読みください**

TABLE OF CONTENTS ENGLISH	PAGE
Product Description	4
Intended Use	4
Regulation & Technical Spec.	4
Limitations	4
Warnings and Precautions	4
Instructions for Use	5
Cleaning	5
Incident Reporting	5
Hazards Associated with Re-Use	5
Symbol Reference Key	18

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ENGLISH

PRODUCT DESCRIPTION

Face shields are personal protective equipment devices that are used for protection of the facial area and associated mucous membranes (eyes, nose, mouth) from splashes, sprays, and spatter of body fluids. Face shields provide a barrier to acutely-expelled aerosols of body fluids and protect a large area of the face.

INTENDED USE

The Summit Medical Face Shield is intended to provide top, side, and front face protection from aerosols and to minimize airborne cross-contamination.

REGULATION & TECHNICAL SPECIFICATION

BSI's PPE for healthcare professionals 2020/403 eye protection.
ANSI Z87.1 Eye protection.

WARNINGS AND PRECAUTIONS

- DO NOT RE-USE OR RE-STERILIZE THIS DEVICE. It is designed for a SINGLE-USE
- This device is supplied clean but is NON-STERILE
- It is recommended that this device is used as part of a PPE regimen
- Care should be taken when putting on and taking off the device to prevent contamination
- Care should be taken if it is necessary to adjust the shield during use, as this may lead to contamination
- Ensure that the shield conforms to the face/head and that the shield does not slip while in use
- Check before use that the shield will not become detached from the elastic headband
- After use, the shield should be carefully removed without contaminating the eyes, mucous membranes, or clothing with potentially infectious materials and discarded
- The product may come into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination
- Dispose of this device after use in accordance with standard hospital protocol
- Do not use five (5) years after date of manufacture
- This face shield is manufactured for COVID-19 protection only
- This face shield is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19

INSTRUCTIONS FOR USE

Follow hospital approved protocol for putting the device on in conjunction with other PPE. Put on the shield, taking care to ensure that it is secure and correctly fitted. To remove, handle via the head band. The shield should be carefully removed without contaminating the eyes, mucous membranes, or clothing with potentially infectious materials and discarded.

CLEANING

This device is non-sterile and is not intended to be disinfected or sterilized.

INCIDENT REPORTING

Any serious incident that has occurred in relation to this device should be reported to the manufacturer.

HAZARDS ASSOCIATED WITH RE-USE

1. Single-use devices have not been validated for re-use.
2. If you re-use a device, you may be held Legally Liable for the safe performance.
3. Cross-contamination and infection risks to patients including transmission of:
 - CJD & Variant CJD
 - Prion Diseases
 - Bacterial Endotoxins
 - Hepatitis B & Hepatitis C
 - Risks posed by HIV and AIDS

Note: Reprocessing may not remove all viable microorganisms. Abnormal proteins associated with prion diseases eg. Creutzfeldt-Jakob disease (CJD) and variant Creutzfeldt-Jakob disease, have been identified in corneal tissue and are very resistant to all conventional methods of decontamination.

4. Device failure through material fatigue or degradation caused by initial use and design.
 - Materials may degrade, becoming unacceptable when compared to original manufacturing criteria
5. Patient injury from device failure and/or chemical burns from residue of decontamination agents absorbed into the materials.

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Summit Medical LLC