

# EZ Face Shield



Our self-assembled full face shield is 9" in length and constructed using a P.E.T. plastic shield, polyurethane foam strip, and adjustable headband for your comfort.

## PROTECT YOUR CUSTOMERS & EMPLOYEES

- Health Care
- Office
- Retail
- Industrial
- Everyday Use


### FOAM STRIP

10 ½ in (L) x 1 in (W) x 1 ¼ in (H)

### ADJUSTABLE BAND

### ANTI-FOG P.E.T. PLASTIC SHIELD

13 in (L) x 9 in (H)

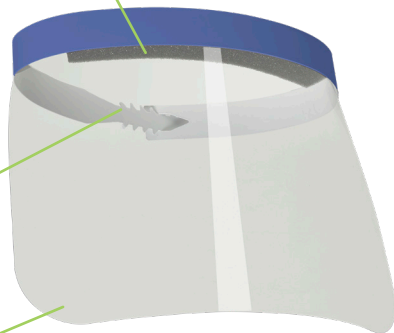

## PRODUCT INFORMATION

### SPECIAL FEATURES

- FDA compliant PPE
- Full face protection
- Proudly made in the USA
- Lightweight and comfortable for long shifts
- Helps protect against splashes, sprays, and bodily fluids
- Adjustable to fit most head sizes better
- Single-Use

## 5 SECOND ASSEMBLY


1. Place adjustable band face down, white side up.



2. Place the shield between the seams only covering half the height of the adjustable band.



3. Peel backing and adhere foam to both the visor and the adjustable band between the seams.

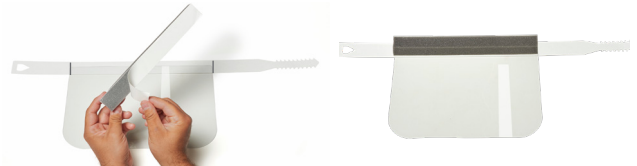

4. Adjust band length using the self locking arrow latch. Readjust to fit if needed.



## ORDERING INFORMATION

PART NUMBER	QTY
CM-5001-EZ	50

Custom labels available for large orders

## CONTACT US

1.888.229.2875

customerservice@innoviamedical.com