INSTRUSAFE

ROUNDERS POINT GUARDS

(EN) ROUNDERS POINT GUARDS	INSTRUCTIONS FOR USE: English
(FR) ROUNDERS POINT GUARDS	MODE D'EMPLOI : Français
(DE) ROUNDERS POINT GUARDS	GEBRAUCHSANWEISUNG: Deutsch
(IT) ROUNDERS POINT GUARDS	ISTRUZIONI PER L'USO: Italiano
(ES) ROUNDERS POINT GUARDS	MODO DE EMPLEO: Español
(JA) ROUNDERS POINT GUARDS	使用説明書: 日本語
(PT) ROUNDERS POINT GUARDS	INSTRUÇÕES DE USO: Português
(NL) ROUNDERS POINT GUARDS	GEbruiksaanwijzing: Nederlands
(SV) ROUNDERS POINT GUARDS	BRUKSANVISNING: Svenska
(DA) ROUNDERS POINT GUARDS	BRUGSANVISNING: Dansk
(FI) ROUNDERS POINT GUARDS	KÄYTTÖOHJEET: Suomi
(EL) ROUNDERS POINT GUARDS	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ: Ελληνικά
(PL) ROUNDERS POINT GUARDS	INSTRUKCJA UŻYTKOWANIA: Polski
(TR) ROUNDERS POINT GUARDS	KULLANMA TALİMATLARI: Türkçe
(RU) ROUNDERS POINT GUARDS	ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ: Русский

(CS) ROUNDERS POINT GUARDS	NÁVOD K POUŽITÍ: Česky
(HU) ROUNDERS POINT GUARDS	HASZNÁLATI UTASÍTÁS: Magyar
(SK) ROUNDERS POINT GUARDS	POKYNY NA POUŽITIE: Slovenčina
(NO) ROUNDERS POINT GUARDS	BRUKSANVISNING: Norsk
(SL) ROUNDERS POINT GUARDS	NAVODILA ZA UPORABO: Slovenski
(BG) ROUNDERS POINT GUARDS	ИНСТРУКЦИИ ЗА УПОТРЕБА: Български
(RO) ROUNDERS POINT GUARDS	INSTRUCTIUNI DE UTILIZARE: Română
(ET) ROUNDERS POINT GUARDS	KASUTUSJUHEND: Eesti
(LV) ROUNDERS POINT GUARDS	LIETOŠANAS INSTRUKCIJA: Latviešu
(HR) ROUNDERS POINT GUARDS	UPUTE ZA UPORABU: Hrvatski
(SR) ROUNDERS POINT GUARDS	UPUTSTVA ZA UPOTREBU: Srpski
(LT) „ROUNDERS POINT GUARDS”	NAUDOJIMO INSTRUKCIJA: lietuvių k.
(CN) ROUNDERS POINT GUARDS	使用说明: 中文
(CH) ROUNDERS POINT GUARDS	使用說明書: 中文
(KO) ROUNDERS POINT GUARDS	사용 지침: 한국어

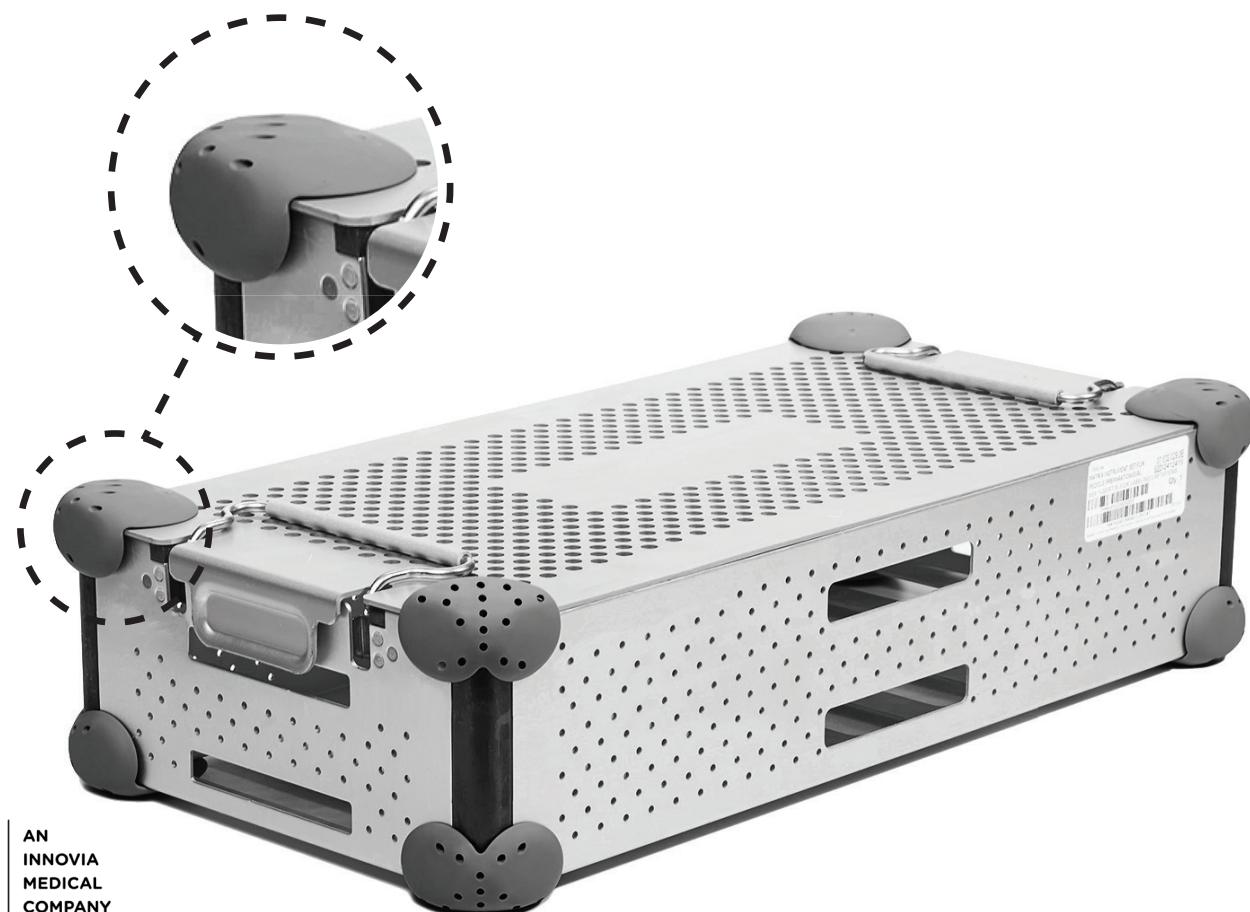
TABLE OF CONTENTS

LANGUAGE	PAGE
English	3
Français	4
Deutsch	5
Italiano	6
Español	7
日本語	8
Português	9
Nederlands	10
Svenska	11
Dansk	12
Suomi	13
Ελληνικά	14
Polski	15
Türkçe	16
Русский	17
Česky	18
Magyar	19
Slovenčina	20
Norsk	21
Slovenski	22
Български	23
Română	24
Eesti	25
Latviešu	26
Hrvatski	27
Srpski	28
Lietuvių k.	29
中文	30
中文	31
한국어	32

READ THIS SECTION BEFORE PLACING PRODUCT INTO SERVICE

INTENDED USE/INDICATIONS FOR USE

InstruSafe® Rounders® Point Guards® are intended to prevent tears in sterilization wrap during storage and transportation. Rounders® Point Guards® are to be used with wrap on tray corners.

INTENDED USER

InstruSafe® Rounders® Point Guards® are intended to be used by healthcare professionals in the operating room and sterile processing departments for transportation, sterilization, and storage of medical devices.

STERILIZATION METHODS

The Rounders Point Guards are compatible with the following sterilization methods:

- Steam-PreVacuum, 270°F (132°C)
- Ethylene Oxide (EO)
- Low Temperature Hydrogen Peroxide

WARNINGS

Use only neutral (6.0-7.5) pH detergents with the Point Guards to avoid damage. A detergent with a highly acidic or highly alkaline pH could permanently damage the Point Guard. Do not use if package is damaged or unintentionally opened prior to use.

CLEANING

1. Manual gross decontamination:

- A. Remove all visible soil and contaminates using a soft bristle brush. The entire Point Guard should be immersed while cleaning to aid in the removal of contaminates and to reduce splashing of detergent on personnel for a minimum of 2 minutes.
- B. Rinse thoroughly for a minimum of 1 minute with clean water to remove all detergent. See rinsing instruction of the detergent label.

2. Ultrasonic clean:

Prepare enzyme wash in an ultrasonic cleaning unit. Place the Point Guard in the detergent and run for a minimum of 10 minutes. Rinse for a minimum of 2 minutes with cold tap water. Visually inspect Point Guard for contaminates. Repeat the cycle if necessary to remove visible contamination.

3. Automated washer:

CYCLE	WATER TEMPERATURE	CLEANING PROCESS
PRE-WASH 1 & 2	COLD TAP WATER	RECIRCULATION TIME: 2 MINUTES
ENZYME WASH	HOT TAP WATER	SOAKING TIME: 4 MINUTES
WASH 1	HEATED AT 65.5°C	RECIRCULATION TIME: 15 MINUTES
RINSE 1 & 2	HOT TAP WATER	RECIRCULATION TIME: 5 MINUTES

Note: After completion of an automatic wash cycle, visually inspect Point Guard for any remaining soil. All visible soil must be removed by hand cleaning, brushing, ultrasonic and additional automatic cycles prior to sending to sterilization.

MAINTENANCE AND INSPECTION

Point Guards were tested for 100 cycles in an automated washer and 100 cycles in an autoclave. Replace Point Guards when inspection reveals deterioration such as: cracking, peeling, flaking, discoloration, mechanical failure, etc.

PACKAGING

Before wrapping the tray ensure the Point Guard is securely placed on the tray.

DISPOSAL

In the event the Rounders® Point Guards® do not pass inspection prior to use or have otherwise been deemed no longer fit for purpose, the devices shall be disposed of in line with local protocol. The method of disposal shall depend on the potential risks of cross-contamination and infection when the need for disposal is identified.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the FDA/competent authority of the Member State in which the user and/or patient is established.

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InstruSafe® and Rounders® Point Guards® are registered trademarks of Summit Medical.
MDSS GmbH
Summit Medical LLC