
The Denver Splint®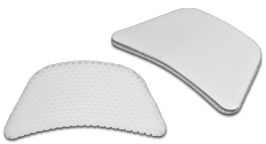

REF	72-3020	Malleable External Nasal Splint, Small Atelle nasale externe malléable, petite Formbare äußere Nasenschiene, klein Tala nasal externa maleável, pequena Pružná externí nosní dlaha, malá Férlula nasal externa maleable, pequena Tutore nasale esterno plasmabile, piccolo Formbar extern nasskiena, liten Verformbare uitwendige neusspalk, Small
REF	72-3021	Malleable External Nasal Splint, Medium Atelle nasale externe malléable, moyenne Formbare äußere Nasenschiene, mittel Tala nasal externa maleável, Média Pružná externí nosní dlaha, střední Tutore nasale esterno plasmabile, medio Formbar extern nasskiena, medium Verformbare uitwendige neusspalk, Medium
REF	72-3022	Malleable External Nasal Splint, Large Atelle nasale externe malléable, grande Formbare äußere Nasenschiene, groß Tala nasal externa maleável, Grande Pružná externí nosní dlaha, velká Férlula nasal externa maleable, grande Tutore nasale esterno plasmabile, grande Formbar extern nasskiena, stor Verformbare uitwendige neusspalk, Large
REF	72-3025	Thermoplastic External Nasal Splint, Small Atelle nasale externe thermoplastique, petite Äußere Thermokunststoff-Nasenschiene, klein Tala nasal externa termoplastică, mică Termoplastică externí nosní dlaha, malá Férlula nasal externa termoplastică, pequena Tutore nasale esterno termoplastică, piccolo Termoplastisk extern nasskiena, liten Thermoplastische uitwendige neusspalk, Small
REF	72-3026	Thermoplastic External Nasal Splint, Medium Atelle nasale externe thermoplastique, moyenne Äußere Thermokunststoff-Nasenschiene, mittel Tala nasal externa termoplastică, Média Termoplastică externí nosní dlaha, střední Férlula nasal externa termoplastică, mediana Tutore nasale esterno termoplastică, medio Termoplastisk extern nasskiena, medium Thermoplastische uitwendige neusspalk, Medium
REF	72-3027	Thermoplastic External Nasal Splint, Custom Cut Atelle nasale externe thermoplastique, découpage personnalisé Äußere Thermokunststoff-Nasenschiene, zuschneidbar Tala nasal externa termoplastică, Corte personalizado Termoplastică externí nosní dlaha, individuální velikost Férlula nasal externa termoplastică, corte personalizado Tutore nasale esterno termoplastică, sus misura Termoplastisk extern nasskiena, skräddarsydd Thermoplastische uitwendige neusspalk, op maat gesneden

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INTENDED USE/INDICATIONS FOR USE

The Denver Splint® is intended to provide external support after rhinoplasty or nasal fractures.

The Denver Splint is indicated to be used postoperatively on rhinoplasty and nasal fracture patients to provide cushioned pressure to reduce edema and to provide external stabilization of bony fragments.

INTENDED USER

The Denver Splint is intended to be used by healthcare professionals, such as, plastic and cosmetic surgeons, facial trauma surgeons, and otolaryngologists.

WARNINGS AND PRECAUTIONS

- Do not apply splint directly to skin- may cause mild sensitization in certain sensitive individuals. Apply paper tape first.
- For Single Use Only. Do not reuse. Reuse of device could result in infection, contamination or device failure which could lead to patient harm.
- Do not resterilize.
- Do not use if package is damaged or unintentionally opened prior to use, or if dorsal pad is compressed.
- Do not use in known cases of severe soft tissue injuries that compromise blood supply to the skin.
- Remove splint prior to X-ray, CT or MRI imaging. Splints may interfere with image quality.

INSTRUCTIONS OF APPLICATION

PREPARATION

****For Thermoplastic Splints: Preheat unit or heat water to 160-180°F (71-82°C)***

- Cleanse and dry skin of nose.
- Use alcohol sponge to wipe nose. Dry skin of nose after wiping.
- Express edema manually from skin.
- Apply a uniform coating of SKIN-PREP® to nasal skin and allow to dry (approximately 30 seconds). If an area intended to be covered is missed, wait until SKIN-PREP has dried then reapply.
- Express edema again.
- Apply overlapping paper tape to nose.

APPLICATION

Malleable Splint
<p>7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. DO NOT TOUCH SKIN WITH DORSAL PAD.</p> <p>8. Remove paper backing from adhesive side of splint.</p> <p>9. Shape metal. Apply adhesive side of splint to tape on the nose. DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</p>
Thermoplastic Splint (Pre-Cut)
<p>7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. DO NOT TOUCH SKIN WITH DORSAL PAD.</p> <p>8. Soften splint in 160-180°F (71-82°C) water for 30 to 60 seconds.</p> <p>9. Remove splint from water using forceps or a tongue depressor to avoid burning fingers.</p> <p>10. Blot briefly on towel. The splint will cool but remain workable for approximately one minute.</p> <p>11. Apply softened splint and mold over the nose.</p> <p>12. If further molding is required, remove splint, resoften in hot water and reapply.</p> <p>13. When in place, the splint should touch only the underlying tape and should not touch the skin. DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</p>

Thermoplastic Splint (Custom Cut)
<p>7. Design desired shape using gauze or paper towel. Check fit to patient.</p> <p>8. Place shape over Thermoplastic Sheet/Flat and cut splint to shape with bandage scissors.</p> <p>9. Soften splint in 160-180°F (71-82°C) water for 30 to 60 seconds.</p> <p>10. Remove splint from water using forceps or a tongue depressor to avoid burning fingers.</p> <p>11. Blot briefly on towel. The splint will cool but remain workable for approximately one minute.</p> <p>12. Apply softened splint and mold over the nose.</p> <p>13. If further molding is required, remove splint, resoften in hot water and reapply.</p> <p>14. When in place, the splint should touch only the underlying tape and should not touch the skin DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</p>

Thermoplastic Splint (Custom Cut)

- Design desired shape using gauze or paper towel. Check fit to patient.
- Place shape over Thermoplastic Sheet/Flat and cut splint to shape with bandage scissors.
- Soften splint in 160-180°F (71-82°C) water for 30 to 60 seconds.
- Remove splint from water using forceps or a tongue depressor to avoid burning fingers.
- Blot briefly on towel. The splint will cool but remain workable for approximately one minute.
- Apply softened splint and mold over the nose.
- If further molding is required, remove splint, resoften in hot water and reapply.
- When in place, the splint should touch only the underlying tape and should not touch the skin **DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.**

REMOVAL

Remove splint in seven to ten days. Recommend removing by spreading bayonet forceps (dull) between skin and tape. Do not tent up skin with removal. Dispose of device as clinical waste in line with local protocol.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the FDA/competent authority of the Member State in which the user and/or patient is established.

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Denver Splint® is a registered trademark of Summit Medical, an Innovia Medical Company. SKIN-PREP® is a registered trademark of Smith & Nephew. VELCRO® is a registered trademark of Velcro Industries B.V. B. A. **SINGLE-USE ONLY.** Not made with natural rubber latex. Non-Sterile. Brevets n° 4,153,051 4,213,452