**SYMBOL REFERENCE TABLE/TABLEAU DE RÉFÉRENCE DES SYMBOLES/ TABELLE ZUR ERKLÄRUNG DER SYMBOLE/TABELA DE REFERÊNCIA DOS SÍMBOLOS/TABLA DE REFERENCIA DE LOS SÍMBOLOS/LEGENDA DEI SIMBOLI**

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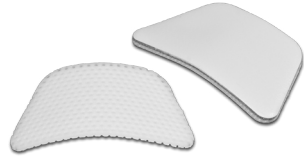
**SHIPPERT MEDICAL**  
**The Denver Splint®**

Series: 1500, 1800, 2000, 3000, 4000, 5000, 5500  
Kits and Non-Kits/Kits et non-kits/Kits und Einzelprodukte/Em Kits e em Separado/Kits y piezas individuales/In kit e in confezioni di soli tutori

Contents/Sommaire/Inhalt/Índice/Contenido/Contenuto	CE		Sterilization/Sterilisation/Estérilização/Estérilização/Sterilizzazione
Protecto™ Tape	CE0086	Aspen Surgical Products, Inc	STERILE EO
Skin-Prep®	CE0086	Smith & Nephew Medical Limited	STERILE R
Webcol™	CE0123	Covidien LLC.	STERILE R
Denver Splint	CE	Summit Medical	
Dorsal pad		Summit Medical	

**NON-KITS DO NOT CONTAIN WEBCOL, SKIN-PREP OR PROTECTO TAPE**  
**LES NON-KITS NE CONTIENNENT PAS DE WEBCOL, SKIN-PREP OU RUBAN PROTECTO**  
**BEI EINZELPRODUKTEN IST KEIN WEBCOL, SKIN-PREP ODER PROTECTO TAPE IM LIEFERUMFANG ENTHALTEN**  
**OS PRODUTOS FORNECIDOS EM SEPARADO NÃO CONTÊM WEBCOL, SKIN-PREP OU FITA ADESIVA PROTECTO**  
**LAS PIEZAS INDIVIDUALES NO CONTIENEN WEBCOL, SKIN-PREP NI ESPARADRAPO PROTECTO**  
**LE CONFEZIONI CONTENUTI SOLO I TUTORI NON INCLUDONO I PRODOTTI WEBCOL, SKIN-PREP O PROTECTO TAPE.**

\*SIZES AVAILABLE: Large, Small/Medium, Petite & Mini  
Patent #s 4,153,051 4,213,452  
Denver Splint® is a registered trademark of Shippert Medical, an Innovia Medical Company  
Skin-Prep® is a registered trademark of Smith & Nephew  
VELCRO® and VELFOAM® are registered trademarks of Velcro Industries B.V. B. A.

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**ESCLUSIVAMENTE MONOUSO.**

## INTENDED USE

The Denver Nasal Splint\* is intended to provide external support nasal surgery or trauma.

## INDICATIONS

For cushioned pressure to reduce edema, external stabilization of bony fragments, protections from moderate external trauma, & masking effect because of small size and appearance.

## CONTRAINDICATIONS

Do not apply directly to skin. Apply paper tape first.

Do not use if package is damaged.

## INSTRUCTIONS OF APPLICATION (KIT & NON-KIT)

### PREPARATION

*\*For series 4000: Preheat unit or heat water to 160-180°F*

- Cleanse and dry skin of nose.
- Use alcohol sponge to wipe nose. Dry skin of nose after wiping.
- Express edema manually from skin
- Apply Skin-Prep® to nasal skin and allow to dry.
- Express edema again.
- Apply overlapping paper tape to nose.

## APPLICATION

Series 1500 Kit & Non-Kit
<p>7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. <b>DO NOT TOUCH SKIN WITH DORSAL PAD.</b></p> <p>8. Remove paper backing from Velcro® (ie. adhesive segment) of splint and apply to paper tape on nose. <b>DO NOT TOUCH SKIN WITH SPLINT.</b> Narrow end of splint should point toward nasofrontal angle.</p> <p>9. Shape metal segment of splint and attach to Velcro.</p> <p>10. Mold by bending wings of splint toward face to make Velcro contact. <b>DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</b></p>

Series 1800 Kit & Non-Kit
<p>7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. <b>DO NOT TOUCH SKIN WITH DORSAL PAD.</b></p> <p>8. Remove paper backing from Velfoam® adhesive segment of splint and apply to paper tape on nose. <b>DO NOT TOUCH SKIN WITH SPLINT.</b> Narrow end of splint should point toward nasofrontal angle.</p> <p>9. Shape metal segment of splint and attach hook Velcro® to Velfoam splint segment.</p> <p>10. Mold by bending wings of splint toward face to make Velcro contact. <b>DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</b></p>

Series 2000 Kit & Non-Kit
<p>7. Trim dorsal pad length to fit splint size. Apply Center dorsal pad vertically to tape on the dorsum of the nose. <b>DO NOT TOUCH SKIN WITH DORSAL PAD.</b></p> <p>8. Remove paper backing from adhesive side of splint.</p> <p>9. Shape metal. Apply adhesive side of splint to tape on the nose. <b>DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</b></p>

Series 3000 Kit & Non-Kit
<p>7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. <b>DO NOT TOUCH SKIN WITH DORSAL PAD.</b></p> <p>8. Shape the aluminum splint as required, and apply to nose. <b>DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</b></p> <p>9. Tape in desired position.</p> <p>10. Splint must be taped because there is no adhesive.</p>

Series 4000 Kit & Non-Kit
<p>8. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. <b>DO NOT TOUCH SKIN WITH DORSAL PAD.</b></p> <p>9. Soften splint in 160-180°F water for 30 to 60 seconds.</p> <p>10. Remove splint from water and blot briefly on towel.</p> <p>11. Apply splint and mold over the nose.</p> <p>12. When in place, the splint should touch only the underlying tape and should not touch the skin.</p>

Series 5000 Kit & Non-Kit
<p>7. Remove paper backing from adhesive segment of Velcro® Loop Splint Segment and apply to paper tape on nose. Narrow end of Velcro Loop Splint Segment should point toward nasofrontal angle.</p> <p>8. Shape metal segment of splint and attach hook Velcro to loop splint segment. Attached dorsal pad will be vertical on dorsum of the nose. Mold by bending wings of splint toward the face to make Velcro contact. <b>DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</b></p>

Series 5500 Kit & Non-Kit
<p>7. Remove paper backing from adhesive side of splint.</p> <p>8. Shape metal. Apply adhesive side of splint to tape on the nose. Attached dorsal pad will be vertical on dorsum of the nose. <b>DO NOT PINCH. DO NOT TOUCH SKIN WITH SPLINT.</b></p>

### REMOVAL

Remove splint in seven to eight days. Recommend removing by spreading bayonet forceps (dull) between skin and tape. Do not tent up skin with removal.

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