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INTENDED USE/INDICATIONS FOR USE
The Denver Split[®] is intended to provide external support after rhinoplasty or nasal fractures.

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ES

INTENDED USER
The Denver Split[®] is intended to be used by healthcare professionals, such as, plastic and cosmetic surgeons, facial trauma surgeons, and otolaryngologists.

CONTRADICATIONS
SKIN-PREP[®] for use on intact skin only.

WARNINGS AND PRECAUTIONS
1. Do not apply splint directly to skin- may cause mild sensitization in certain sensitive individuals.
2. Apply protective tape first.

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ES

SKIN-PREP[®] for use on intact skin only.

CONTRAINDICATIONS
SKIN-PREP[®] for use on intact skin only.

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ES

2. For Single Use Only. Do not reuse. Reuse of device could result in infection, contamination or device failure which could lead to patient harm.

3. Do not sterilize.

4. Do not use if the device is damaged or unintentionally opened prior to use, or if dorsal pad is compromised.

5. Do not use in known cases of severe soft tissue injuries that compromise blood supply to the skin.

6. Remove protective splint prior to X-ray, CT or MRI imaging. Splints may interfere with image quality.

7. SKIN-PREP[®] - Warning: causes serious eye irritation. Avoid contact with eyes. In the case of accidental contact, flush eyes well with water. Should redness or other signs of irritation appear, discontinue use. Warning: may cause drowsiness and dizziness. Danger: Highly flammable liquid and vapor. Storage: Keep dry and away from sun; upper temperature limit of 25°C/77°F. Protecto Tape - Skin should be clean, dry, and free of oils to assure good adhesion.

ES

APPLICATION INSTRUCTIONS

***For series 4000 and Thermoplastic Sheets/Flats: Preheat unit or heat water to 160-180° F (71-82°C)**

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ES

1. Cleanse and dry skin.

2. Remove paper backing from tape. Dry skin of nose after wiping.

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ES

3. Express edema manually from skin.

4. Apply a uniform coating of SKIN-PREP[®] to nasal skin and allow to dry (approximately 30 seconds).

5. If an area intended to be covered is missed, wait until SKIN-PREP has dried then re-apply.

6. Express edema again.

7. Apply overlapping paper tape to nose.

8. Protecto Tape - Skin should be clean, dry, and free of oils to assure good adhesion.

ES

APPLICATION

Série 1500 Kit & Non-Kit

PRÉPARATION

PREPARACIÓN

PREPARAÇÃO

PRÉPARAÇAO

ES

7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. **DO NOT TOUCH SKIN WITH DORSAL PAD.**

8. Remove paper backing from adhesive segment of Velcro® and apply to paper tape on nose. **DO NOT TOUCH SKIN WITH SPLINT. Narrow end of splint should point toward nasofrontal angle.**

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ES

9. Shape metal segment of splint and attach to Velcro.

10. Mold by bending wings of splint toward face to make Velcro contact. **DO NOT PINCH.**

ANwendung

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ES

APPLICATION

Série 2000 Kit & Non-Kit

PRÉPARATION

PREPARACIÓN

PREPARAÇÃO

PRÉPARAÇAO

ES

7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. **DO NOT TOUCH SKIN WITH DORSAL PAD.**

8. Remove paper backing from adhesive segment of Velcro® and apply to paper tape on nose. **DO NOT TOUCH SKIN WITH SPLINT. Narrow end of splint should point toward nasofrontal angle.**

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ES

9. Shape metal segment of splint and attach to Velcro.

10. Mold by bending wings of splint toward face to make Velcro contact. **DO NOT PINCH.**

ANwendung

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ES

APPLICATION

Série 4000 Kit & Non-Kit

PRÉPARATION

PREPARACIÓN

PREPARAÇÃO

PRÉPARAÇAO

ES

7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. **DO NOT TOUCH SKIN WITH DORSAL PAD.**

8. Remove paper backing from adhesive segment of Velcro® and apply to paper tape on nose. **DO NOT TOUCH SKIN WITH SPLINT. Narrow end of splint should point toward nasofrontal angle.**

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ES

9. Shape metal segment of splint and attach to Velcro.

10. Mold by bending wings of splint toward face to make Velcro contact. **DO NOT PINCH.**

ANwendung

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ES

APPLICATION

Série 4000 Kit & Non-Kit

PRÉPARATION

PREPARACIÓN

PREPARAÇÃO

PRÉPARAÇAO

ES

7. Trim dorsal pad length to fit splint size. Apply center dorsal pad vertically to tape on the dorsum of the nose. **DO NOT TOUCH SKIN WITH DORSAL PAD.**

8. Remove paper backing from adhesive segment of Velcro® and apply to paper tape on nose. **DO NOT TOUCH SKIN WITH SPLINT. Narrow end of splint should point toward nasofrontal angle.**

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ES

9. Shape metal segment of splint and attach to Velcro.

10. Mold by bending wings of splint toward face to make Velcro contact. **DO NOT PINCH.**

ANwendung

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ES