
INSTRUSAFE

ROUNDERS® POINT GUARDS®

EN ROUNDERS POINT GUARDS **INSTRUCTIONS FOR USE**
FR ROUNDERS POINT GUARDS **MODE D'EMPLOI :**
DE ROUNDERS POINT GUARDS **GEBRAUCHSANWEISUNG:**
IT ROUNDERS POINT GUARDS **ISTRUZIONI PER L'USO:**
ES ROUNDERS POINT GUARDS **MODO DE EMPLEO:**
JA ROUNDERS POINT GUARDS **使用説明書:**
PT-BR ROUNDERS POINT GUARDS **INSTRUÇÕES DE USO:**
NL ROUNDERS POINT GUARDS **GEbruIKSAANWIJZING:**
SV ROUNDERS POINT GUARDS **BRUKSANVISNING:**
DA ROUNDERS POINT GUARDS **BRUGSANVISNING:**
FI ROUNDERS POINT GUARDS **KÄYTTÖOHJEET:**
EL ROUNDERS POINT GUARDS **ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ:**
PL ROUNDERS POINT GUARDS **INSTRUKCJA UŻYTKOWANIA:**
TR ROUNDERS POINT GUARDS **KULLANMA TALİMATLARI:**
RU ROUNDERS POINT GUARDS **ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ:**

CS ROUNDERS POINT GUARDS **NÁVOD K POUŽITÍ:**
HU ROUNDERS POINT GUARDS **HASZNÁLATI UTASÍTÁS:**
SK ROUNDERS POINT GUARDS **POKYNY NA POUŽITIE:**
NO ROUNDERS POINT GUARDS **BRUKSANVISNING:**
SL ROUNDERS POINT GUARDS **NAVODILA ZA UPORABO:**
BG ROUNDERS POINT GUARDS **ИНСТРУКЦИИ ЗА УПОТРЕБА:**
RO ROUNDERS POINT GUARDS **INSTRUCȚIUNI DE UTILIZARE:**
ET ROUNDERS POINT GUARDS **KASUTUSJUHEND:**
LV ROUNDERS POINT GUARDS **LIETOŠANAS INSTRUKCIJA:**
HR ROUNDERS POINT GUARDS **UPUTE ZA UPORABU:**
SR ROUNDERS POINT GUARDS **UPUTSTVA ZA UPOTREBU:**
LT „ROUNDERS POINT GUARDS“ **NAUDOJIMO INSTRUKCIJA:**
zh-CN ROUNDERS POINT GUARDS **使用说明:**
zh ROUNDERS POINT GUARDS **使用説明書:**
KO ROUNDERS POINT GUARDS **사용 지침:**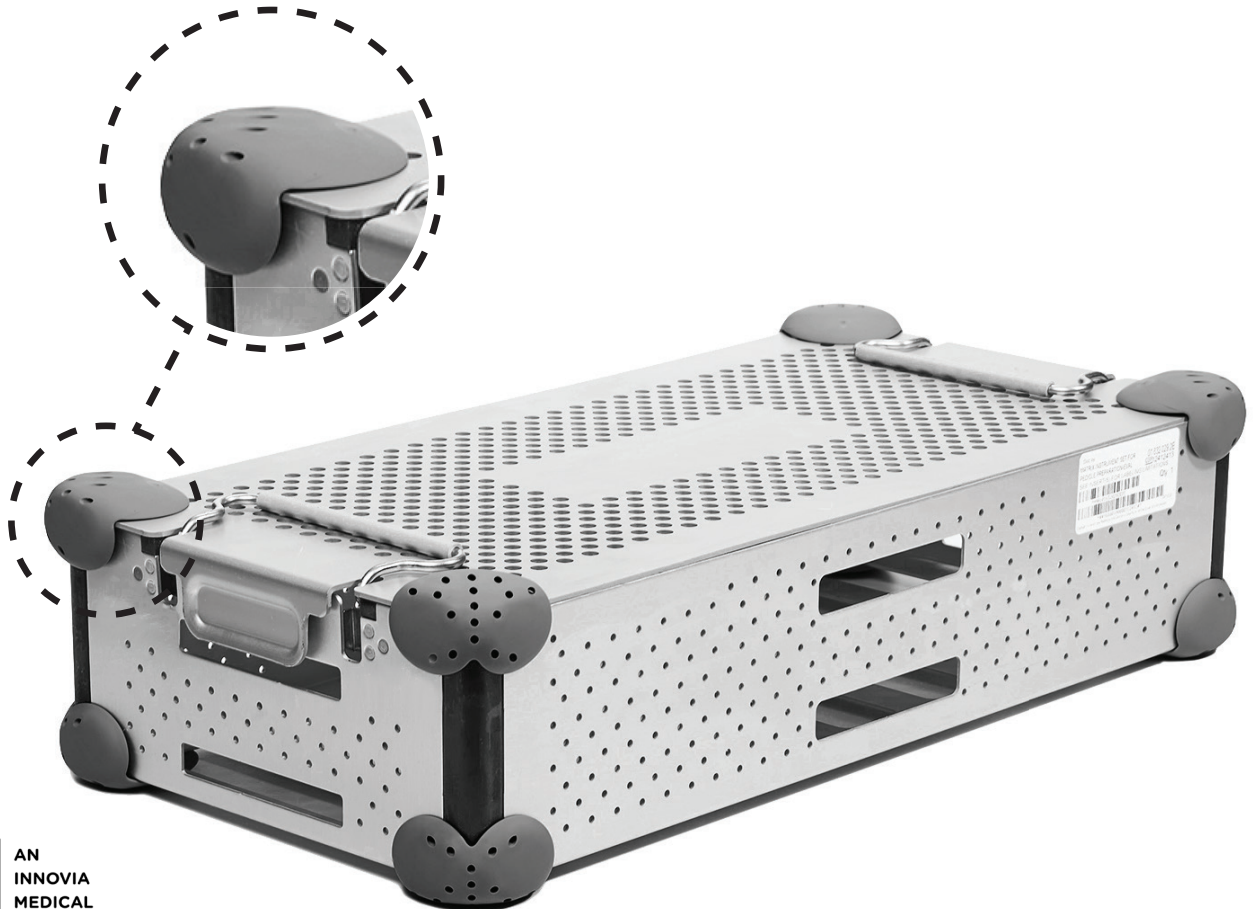

For the full instructions for use visit:
Pour le mode d'emploi complet, visitez le site :
Die vollständige Gebrauchsanweisung finden Sie unter:
Per le istruzioni per l'uso complete, visitare:
Para ver las instrucciones completas de uso, visite:
Para as instruções completas de utilização visite:
Ga voor de volledige gebruiksaanwijzing naar:
För att få tillgång till hela bruksanvisningen, besök:
Se hele brugsanvisningen her:
Täydelliset käyttöohjeet ovat osoitteessa:
Για πλήρεις οδηγίες χρήσης, επισκεφτείτε:
Pełną instrukcję użytkowania można znaleźć na stronie:
Kullanma talimatlarının tamamı için şu adresi ziyaret edin:
Для получения полной инструкции по эксплуатации посетите:
Pro úplný návod k použití navštivte:
A teljes használati utasításért látogasson el ide:
Úplné pokyny na použitie nájdete na:
For en komplett bruksanvisning, gå til:
Za celotna navodila obiščite:
За пълните инструкции за употреба посетете:
Pentru instrucțiunile de utilizare complete, accesați:
Täieliku kasutusjuhendi leiate aadressilt:
Lietošanas instrukcijas pilna versija ir pieejama šajā tīmekļa vietnē:
Potpune upute za upotrebu pronađite pod:
Kompletna uputstva potražite na adresi:
Išsamią naudojimo instrukciją rasite:
有关完整的使用说明，请访问：
有關完整的使用說明，請瀏覽：
전체 사용 지침은 다음 사이트를 방문하십시오.
使用説明書の詳細については、以下をご覧ください。

www.instrusafe.com/ifus

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READ THIS SECTION BEFORE PLACING PRODUCT INTO SERVICE

INTENDED USE/INDICATIONS FOR USE

InstruSafe® Rounders® Point Guards® are intended to prevent tears in sterilization wrap during storage and transportation. Rounders® Point Guards® are to be used with wrap on tray corners.

INTENDED USER

InstruSafe® Rounders® Point Guards® are intended to be used by healthcare professionals in the operating room and sterile processing departments for transportation, sterilization, and storage of medical devices.

STERILIZATION METHODS

The Rounders Point Guards are compatible with the following sterilization methods:

- Steam-PreVacuum, 279°F (137°C)
- Ethylene Oxide (EO)
- Low Temperature Hydrogen Peroxide

WARNINGS

Do not use if package is damaged or unintentionally opened prior to use.

CLEANING

1. Manual gross decontamination:

- A. Remove all visible soil and contaminants using a soft bristle brush. The entire Point Guard should be immersed while cleaning to aid in the removal of contaminants and to reduce splashing of detergent on personnel for a minimum of 2 minutes.
- B. Rinse thoroughly for a minimum of 1 minute with clean water to remove all detergent. See rinsing instruction of the detergent label.

2. Ultrasonic clean:

Prepare enzyme wash in an ultrasonic cleaning unit. Place the Point Guard in the detergent and run for a minimum of 10 minutes. Rinse for a minimum of 2 minutes with cold tap water. Visually inspect Point Guard for contaminants. Repeat the cycle if necessary to remove visible contamination.

3. Automated washer:

CYCLE	WATER TEMPERATURE	CLEANING PROCESS
PRE-WASH 1 & 2	COLD TAP WATER	RECIRCULATION TIME: 2 MINUTES
ENZYME WASH	HOT TAP WATER	SOAKING TIME: 4 MINUTES
WASH 1	HEATED AT 65.5°C	RECIRCULATION TIME: 15 MINUTES
RINSE 1 & 2	HOT TAP WATER	RECIRCULATION TIME: 5 MINUTES

Note: After completion of an automatic wash cycle, visually inspect Point Guard for any remaining soil. All visible soil must be removed by hand cleaning, brushing, ultrasonic and additional automatic cycles prior to sending to sterilization.

MAINTENANCE AND INSPECTION

Point Guards were tested for 100 cycles in an automated washer and 100 cycles in an autoclave. Replace Point Guards when inspection reveals deterioration such as: cracking, peeling, flaking, discoloration, mechanical failure, etc.

PACKAGING

Before wrapping the tray ensure the Point Guard is securely placed on the tray.

DISPOSAL

In the event the Rounders® Point Guards® do not pass inspection prior to use or have otherwise been deemed no longer fit for purpose, the devices shall be disposed of in line with local protocol. The method of disposal shall depend on the potential risks of cross-contamination and infection when the need for disposal is identified.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the FDA/competent authority of the Member State in which the user and/or patient is established.

InstruSafe® and Rounders® Point Guards® are registered trademarks of Summit Medical.

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