




INTRUSAFE

XL TRANSPORT CONTAINER

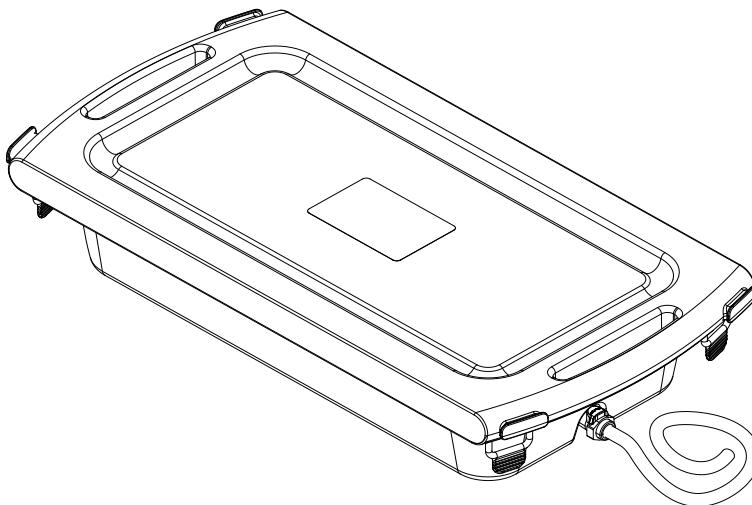

- | | | |
|---------|------------------------|--|
| (EN) | XL TRANSPORT CONTAINER | INSTRUCTIONS FOR USE: English |
| (FR) | XL TRANSPORT CONTAINER | MODE D'EMPLOI: Français |
| (DE) | XL TRANSPORT CONTAINER | GEBRAUCHSANWEISUNG: Deutsch |
| (IT) | XL TRANSPORT CONTAINER | ISTRUZIONI PER L'USO: Italiano |
| (ES) | XL TRANSPORT CONTAINER | MODO DE EMPLEO: Español |
| (JA) | XL TRANSPORT CONTAINER | 使用説明書: 日本語 |
| (PT-BR) | XL TRANSPORT CONTAINER | INSTRUÇÕES DE UTILIZAÇÃO: Português |
| (NL) | XL TRANSPORT CONTAINER | GEbruIKSAANWIJZING: Nederlands |
| (SV) | XL TRANSPORT CONTAINER | ANVÄNDARINSTRUKTIONER: Svenska |
| (DA) | XL TRANSPORT CONTAINER | BRUGSANVISNING: Dansk |
| (FI) | XL TRANSPORT CONTAINER | KÄYTTÖOHJEET: Suomi |
| (EL) | XL TRANSPORT CONTAINER | ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ: Ελληνικά |
| (PL) | XL TRANSPORT CONTAINER | INSTRUKCJA UŻYTKOWANIA: Polski |
| (TR) | XL TRANSPORT CONTAINER | KULLANMA TALİMATLARI: Türkçe |
| (RU) | XL TRANSPORT CONTAINER | ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ: Русский |
| (CS) | XL TRANSPORT CONTAINER | NÁVOD K POUŽITÍ: Český |
| (HU) | XL TRANSPORT CONTAINER | HASZNÁLATI ÚTMUTATÓ: Magyar |
| (SK) | XL TRANSPORT CONTAINER | POKYNY NA POUŽITIE: Slovenčina |
| (NO) | XL TRANSPORT CONTAINER | BRUKSANVISNING: Norsk |
| (SL) | XL TRANSPORT CONTAINER | NAVODILA ZA UPORABO: Slovenski |
| (BG) | XL TRANSPORT CONTAINER | ИНСТРУКЦИИ ЗА УПОТРЕБА: Български |
| (RO) | XL TRANSPORT CONTAINER | INSTRUCȚIUNI DE UTILIZARE: Română |
| (ET) | XL TRANSPORT CONTAINER | KASUTUSJUHEND: Eesti |
| (LV) | XL TRANSPORT CONTAINER | LIETOŠANAS INSTRUKCIJA: Latviešu |
| (HR) | XL TRANSPORT CONTAINER | UPUTE ZA UPORABU: Hrvatski |
| (SR) | XL TRANSPORT CONTAINER | UPUTSTVO ZA UPOTREBU: Srpski |
| (LT) | XL TRANSPORT CONTAINER | NAUDOJIMO INSTRUKCIJA: lietuvių k. |
| (ZH-CN) | XL TRANSPORT CONTAINER | 使用说明: 中文 |
| (ZH) | XL TRANSPORT CONTAINER | 請參閱使用說明書: 中文 |
| (KO) | XL TRANSPORT CONTAINER | 사용 지침: 한국어 |

For the full instructions for use visit:
Pour le mode d'emploi complet, visitez le site :
Die vollständige Gebrauchsanweisung finden Sie unter:
Per le istruzioni per l'uso complete visitare:
Para ver las instrucciones de uso completas, visite:
使用説明書の詳細については、以下をご覧ください。
Para as instruções completas de utilização visite:
Ga voor de volledige gebruiksaanwijzing naar:
För att få tillgång till hela bruksanvisningen, besök:
Se hele brugsanvisningen her:
Täydelliset käyttöohjeet ovat osoitteessa:
Για πλήρεις οδηγίες χρήσης, επισκεφτείτε:
Pełną instrukcję użytkowania można znaleźć na stronie:
Kullanma talimatlarının tamamı için şu adresi ziyaret edin:
Для получения полных инструкций по эксплуатации посетите:
Pro úplný návod k použití navštivte:
A teljes használati utasításért látogasson el ide:
Úplné pokyny na použitie nájdete na:
For en komplett bruksanvisning, gå til:
Za celotna navodila obiščite:
За пълните инструкции за употреба посетете:
Pentru instrucțiunile de utilizare complete, accesați:
Täieliku kasutusjuhendi leiate aadressilt:
Lietošanas instrukcijas pilna versija ir pieejama šajā tīmekļa vietnē:
Potpuno upute za upotrebu pronađite pod:
Kompletna uputstva potražite na adresi:
Išsamią naudojimo instrukciją rasite:
有关完整的使用说明，请访问：
有關完整的使用說明，請瀏覽：
전체 사용 지침은 다음 사이트를 방문하십시오:



www.instrusafe.com/ifus

If you have any questions, please contact our customer service representatives at **1.888.229.2875**. / Pour toute question, veuillez contacter nos représentants du service clientèle au **1.888.229.2875**. / Wenn Sie Fragen haben, wenden Sie sich an unser Kundendienst unter **1.888.229.2875**. / In caso di domanda, contattate gli addetti all'assistenza clienti al numero **1.888.229.2875**. / Si tiene alguna pregunta, póngase en contacto con nuestros representantes de servicio de atención al cliente al **1.888.229.2875**. / ご不明な点がございましたら、以下のカスタマーサービス担当者までお問い合わせください **1.888.229.2875** / Se você tiver alguma dúvida, entre em contato com nossos representantes de atendimento ao cliente pelo telefone **1.888.229.2875**. / Neem voor vragen contact op met onze medewerkers van de klantenservice op **1.888.229.2875**. / Om du har frågor, var god kontakta våra kundtjänstrepresentanter på telefon **1.888.229.2875**. / Hvis du har spørgsmål, bedes du kontakte vores kundeservicepræsentanter på **1.888.229.2875**. / Jos sinulla on kysyttävää, ota yhteyttä asiakaspalveluumme puhelinnumerossa **1.888.229.2875**. / Αν έχετε ερωτήσεις, επικοινωνήστε με τους αντιπροσώπους της εξυπηρέτησης πελατών μας στο **1.888.229.2875**. / Wszelkie pytania należy kierować do przedstawicieli działu obsługi klienta pod numerem telefonu **1.888.229.2875**. / Herhangi bir sorunuz varsa, lütfen şu numaradan müşteri hizmetleri temsilcilerimizi arayın: **1.888.229.2875**. / Если у вас есть какие-либо вопросы, свяжитесь с представителями нашей службы поддержки клиентов по телефону **1.888.229.2875**. / Máte-li nějaké dotazy, obraťte se na naše zástupce služeb pro zákazníky na čísle **1.888.229.2875**. / Ha bármilyen kérdése van, kérjük, vegye fel a kapcsolatot ügyfélszolgálati képviselőinkkel az telefonszámon **1.888.229.2875**. / Ak máte akékoľvek otázky, obráťte sa na našich zástupcov zákazníckeho servisu na čísle **1.888.229.2875**. / Hvis du har noen spørsmål, ta kontakt med våre kundeservicepræsentanter på **1.888.229.2875**. / Če imate kakršna koli vprašanja, se obrnite na naše predstavnike službe za stranke na **1.888.229.2875**. / Ако имате някакви въпроси, моля, свържете се с нашите представители за обслужване на клиенти на **1.888.229.2875**. / Dacă aveți întrebări, vă rugăm să contactați reprezentanții departamentului nostru de relații clienți la **1.888.229.2875**. / Kūsimuste korral võtke ühendust meie klienditeenindusega **1.888.229.2875**. / Ja rodas jautājumi, lūdzu, sazinieties ar mūsu klientu apkalpošanas nodaļas pārstāvjiem pa tālruni **1.888.229.2875**. / Ako imate bilo kakva pitanja, nazovite predstavnika naše korisničke službe na **1.888.229.2875**. / Ako imate pitanja, obratite se predstavnicima korisničke podrške na **1.888.229.2875**. / Jei turite klausimų, susisiekite su mūsų klientų aptarnavimo atstovais telefonu **1.888.229.2875**. / 如有任何疑问, 请致电 **1.888.229.2875** 与我们的客户服务代表联系。 / 如有任何疑問, 請撥打電話 **1.888.229.2875** 與我們的客戶服務代表聯絡。 / 질문이 있는 경우, **1.888.229.2875**의 고객 서비스 대표번호로 문의하십시오.

READ THIS SECTION BEFORE PLACING PRODUCT INTO SERVICE

INTENDED USE/ INDICATIONS FOR USE

InstruSafe® XL Transport Containers are intended to contain and protect reusable medical devices during transport from the operating room to the sterile processing department.

The InstruSafe® XL Transport Containers are used to collect used instruments and materials for transportation. The transport container can be used in a washer-disinfector and withstands repeated washing.

INTENDED USER

The InstruSafe® XL Transport Containers are intended to be used by healthcare professionals in the operating room and sterile processing departments for transportation of medical devices.

MATERIAL

The container is reusable and made of high temperature plastic. It is highly durable against heat and chemicals.

DESCRIPTION

The container consists of a lid and base, silicone gasket and latches. The gasket and latches can be removed to help facilitate cleaning and can be easily replaced. The container base holds the instruments and materials for transportation.

LID REMOVAL

Release the (4) silicone latches and remove the lid.

CONTAINER USE

1. Set container on stable surface
2. Add cleaning solution per the manufacturer's IFU to the container
3. Place instruments / materials in container for transportation
4. Place lid on container and engage (4) Silicone latches
5. Transport container
6. *For IN-PIC-20-D: To attach hose, press drain insert into coupling to open the valve
7. *For IN-PIC-20-D: To remove hose and close the valve, depress button and remove drain insert from coupling
8. *For IN-PIC-20-D: Drain should be flushed with water and air to remove any residue
9. Open lid slowly to remove instruments / materials
10. Process instruments / materials per the manufacturer's IFU

WARNINGS

1. **DO NOT** autoclave
2. **DO NOT** use as a sterile barrier
3. **DO NOT** drop as this may cause damage to the instruments / materials inside or damage the container resulting in possible leakage
4. **DO NOT** shake the container with liquids inside. This can cause damage to the contents or container leakage

Life Cycle: **DO NOT** use if container is cracked, broken or missing the silicone gasket or latches

5. Clean and inspect prior to use. Container may be reused until mechanical failure occurs
6. Be cautious of pinch points while lid is being opened and closed to avoid harm to fingers and hands
7. Be cautious of container weight with liquid added
8. **DO NOT** wash container with a hard brush. This can cause damage to the container
9. **DO NOT** use if package is damaged or unintentionally opened prior to use
10. Remove cover before automated washing
11. **DO NOT** stack if optional drain is installed
12. **DO NOT** stretch the silicone gasket


MANUAL CLEANING

1. Use a neutral (6.0 - 8.5) pH detergent
2. Remove all visible soil and contaminants using a soft brush
3. Immerge the container while cleaning to aid in the removal of contaminants and reduce splashing of detergent on personnel
4. Rinse thoroughly with clean water to remove detergent

AUTOMATED WASHER CLEANING


1. Container is safe for use in automated washers and can withstand up to 230°F

Gasket and latches can be replaced without the use of tools. Gasket Replacement

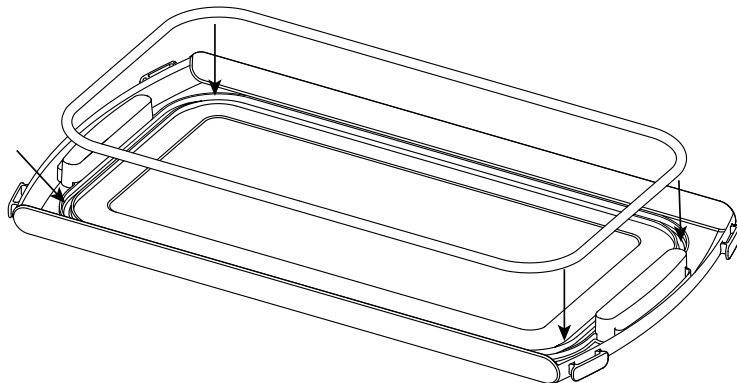

1. **DO NOT** stretch the silicone gasket
2. To insert, align gasket and press down

Latch Replacement

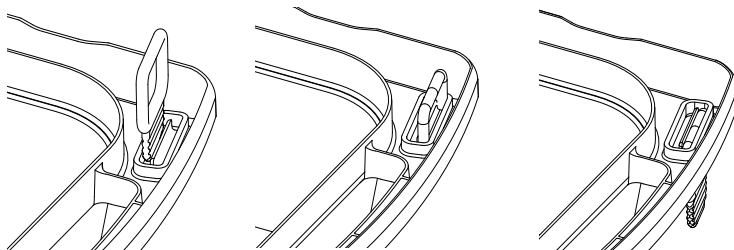

1. **Insert on top**
2. **Position latch over groove**
3. **Pull latch down into the groove**













DISPOSAL

In the event the InstruSafe® XL Transport Containers do not pass inspection prior to use or have otherwise been deemed no longer fit for purpose, the devices shall be disposed of in line with local protocol. The method of disposal shall depend on the potential risks of cross-contamination and infection when the need for disposal is identified.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the FDA/competent authority of the Member State in which the user and/or patient is established.

InstruSafe® is a registered trademark of Summit Medical.
InstruSafe® est une marque déposée de Summit Medical.
InstruSafe® ist eine eingetragene Marke von Summit Medical.
InstruSafe® è un marchio registrato di Summit Medical.
InstruSafe® es una marca registrada de Summit Medical.
InstruSafe® は、Summit Medicalの登録商標です。
InstruSafe® é uma marca registrada da Summit Medical.
InstruSafe® is een geregistreerd handelsmerk van Summit Medical.
InstruSafe® är ett registrerat varumärke som tillhör Summit Medical.
InstruSafe® er et registreret varemærke tilhørende Summit Medical.
InstruSafe® on Summit Medical in rekisteröity tavaramerkki.
Το InstruSafe® είναι σήμα κατατεθέν της Summit Medical.
InstruSafe® jest zastrzeżonym znakiem towarowym firmy Summit Medical.
InstruSafe®, bir Summit Medical tescilli ticari markasıdır.
InstruSafe® является зарегистрированной торговой маркой Summit Medical.
InstruSafe® je registrovaná obchodní značka společnosti Summit Medical.
Az InstruSafe® a Summit Medical bejegyzett védjegye.
InstruSafe® je registrovaná obchodná známka spoločnosti Summit Medical.
InstruSafe® er et registrert varemerke til Summit Medical.
InstruSafe® je registrirana blagovna znamka podjetja Summit Medical.
InstruSafe® е регистрирана търговска марка на Summit Medical.
InstruSafe® este o marcă înregistrată a Summit Medical.
InstruSafe® on ettevõtte Summit Medical registreeritud kaubamärk.
InstruSafe® ir uzņēmuma Summit Medical reģistrēta preču zīme.
InstruSafe® je registrirani trgovački znak tvrtke Summit Medical.
InstruSafe® je registrovani žig kompanije Summit Medical.
„InstruSafe®“ yra registruotasis bendrovės „Summit Medical“ prekių ženklas.
InstruSafe® 是 Summit Medical 的注册商标。
InstruSafe® 是 Summit Medical 的註冊商標。
InstruSafe® 는 Summit Medical의 등록 상표입니다.

Symbol Reference Key / Définition des symboles / Symbollegende / Legenda dei simboli / Definición de los símbolos / 記号参照キー / Chave de referência de símbolo / Legenda / Symbol Referensnyckel / Symbolreferencenagle / Symbolien merkitys / Υπόμνημα συμβόλων / Objaśnienie symboli / Sembol Referans Anahtarları / Расшифровка символов / Použité symboly / Szimbólumok magyarázata / Referenčný kľúč k symbolom / Symbolreferansenøkkel / Služba za pomoč strankam / Легенда на символите / Tastá symbol de referință / Sümboleit legend / Simbolu skaidrojums / Referentni ključ simbola / Legenda simbola / Simbolių paaiškinimas / 符号参考图例 / 符號參考圖例 / 기호 참조 키	
	Consult instructions for use / Consulter le mode d'emploi / Gebrauchsanweisung beachten / Consultare le istruzioni per l'uso / Consulte el modo de empleo 使用説明書を参照してください / Consulte as instruções de uso / Raadpleeg de gebruiksaanwijzing / Se bruksanvisningen / Se brugsanvisning / Lue käyttöohjeet / Συμβουλευθείτε τις οδηγίες χρήσης / Należy zapoznać się z instrukcją użytkowania / Kullanna talimatlarina basyvrur / См. инструкции по эксплуатации / Přečtěte si návod k použití / Tekintse át a használati útmutatót / Pozrite návod na použitie / Se bruksanvisning / Preglejte navodila za uporabo / Направте справка в инструкциите за употреба / Consultați instrucțiunile de utilizare / Vaadake kasutusjuhendit / Skatīt lietošanas instrukciju / Pročítajte upute za uporabu / Pogledajte uputstvo za upotrebu / Žr. naudojimo instrukciją / 请参阅使用说明书 / 請參閱使用說明書 / 사용 지침 참조
	Quantity / Quantité / Menge / Quantità / Cantidad / 數量 / Quantidade / Aantal / Antal / Antal / Määrä / Ποσότητα / Ilość / Miktar / Количество / Množství / Mennyiség / Množstvo / Mengde / Količina / Количество / Cantitate / Kogus / Daudzums / Količina / Količina / Kiekis / 量 / 數量 / 수량
	Catalogue number / Référence catalogue / Bestellnummer / Numero catalogo / Número de catálogo / カタログ番号 / Número de catálogo / Catalogusnummer / Katalognummer / Katalognummer / Luettelonumero / Αριθμός καταλόγου / Numer katalogowy / Katalog numarası / Каталогный номер / Katalogové číslo / Katalógusszám / Katalógové číslo / Katalognummer / Kataloška številka / Каталоген номер / Număr de catalog / Katalooginumber / Kataloga numurs / Kataloški broj / Kataloški broj / Katalog numeris / 目录号 / 目錄號 / 카탈로그 번호
	Batch Code / Code de lot / Chargennummer / Codice lotto / Código de lote / バッチコード / Código do lote / Batchcode / Satsnummer / Partikode / Eränumero / Κωδικός παρτίδας / Kod partii / Parti Kodu / Код партии / Kód sarže / Gyártási kód / Kód sarže / Partikode / Koda serije / Код на партида / Cod lot / Partii kood / Sērijas numurs / Šifra serije / Kód serije / Partijos kodas / 批号 / 批號 / 배치 코드
	Date of manufacture / Date de fabrication / Herstellungsdatum / Data di fabbricazione / Fecha de fabricación / 製造日 / Data de fabricaço / Productiedatum / Tillverkningsdatum / Fremstillingsdato / Valmistuspäivä / Ημερομηνία κατασκευής / Data produkcyj / Üretim tarihi / Дата производства / Datum výroby / Gyártás dátuma / Datum výroby / Produktionsdato / Datum proizvodnje / Дата на производство / Data producției / Tootmiskuupäev / Izgatavošanas datums / Datum proizvodnje / Datum proizvodnje / Pagaminimo data / 生产日期 / 生產日期 / 제조일
	Non-sterile / Non stérile / Nicht steril / Non sterile / No estéril / 非滅菌 / Não esterilizado / Niet-steriel / Ikke-steril / Ei sterili / Μη αποστειρωμένο / Niesterylny / Steril deǵildir / Нестерильно / Nesterilni / Nem steril / Nesterilné / Ikke steril / Nesterilno / Нестерилно / Nesterile / Mittesteriilne / Nesterils / Nesterilno / Nesterilno / Nesterilu / 非灭菌 / 非滅菌 / 비-멸균
	Manufacturer / Fabricant / Hersteller / Produttore / Fabricante / 製造元 / Fabricante / Fabrikant / Tillverkare / Producent / Valmistaja / Κατασκευαστής / Producent / Üretici / Производитель / Výrobce / Gyártó / Výrobca / Produsent / Proizvajalec / Производител / Producător / Tootja / Ražotājs / Proizvođač / Proizvođač / Gamintojas / 製造商 / 製造商 / 제조사
	Do not use if package is damaged / Ne pas utiliser si l'emballage est endommagé / Beschädigte Verpackungen nicht verwenden / Non utilizzare in caso di imballaggio danneggiato / No usar si el envase está dañado / 梱包が破損している場合は使用しないこと / Não use se a embalagem estiver danificada / Niet gebruiken als de verpakking beschadigd is / Får inte användas om förpackningen är skadad / Må ikke bruges, hvis emballagen er beskadiget / Älä käytä, jos pakkaus on vahingoittunut / Μην χρησιμοποιείτε αν η συσκευασία είναι κατεστραμμένη / Nie używać, jeśli opakowanie jest uszkodzone / Ambalaj hasarlıysa kullanmayın / Не использовать, если упаковка повреждена / Nepoužívejte, pokud je obal poškozen / Ne használja, ha a csomagolás sérült / Nepoužívejte, ak je obal poškodený / Ikke bruk hvis pakningen er skadet / Ne uporabljajte, če je paket poškodovan / Да не се използва, ако опаковката е повредена / A nu se utiliza dacă ambalajul este deteriorat / Mitte kasutada, kui pakend on viga saanud / Nelietot, ja iepakojums ir bojāts / Nemojte upotrebljavati ako je ambalaža oštećena / Ne koristite ako je pakovanje oštećeno / Nenaudokite, jei pakuotė pažeista / 如果包装破损, 请勿使用 / 如果包裝損壞, 請勿使用 / 포장이 손상된 경우 사용하지 마십시오.
	Medical Device / Dispositif médical / Medizinprodukt / Dispositivo medico / Dispositivo médico / 醫療機器 / Dispositivo Médico / Medisch hulpmiddel / Medicinski aparat / Medicinska udstyr / Läkintälaite / Ιατρική συσκευή / Wyrób medyczny / Tibbi Cihaz / Медицинское изделие / Zdravotnický prostředek / Orvostechikai eszköz / Zdravotnička pomôcka / Medicinsk utstyr / Medicinska naprava / Медицински изделие / Dispozitiv medical / Meditsiiniseade / Medicínska ierice / Medicinski uređaj / Medicinski uredaj / Medicinos prietaisas / 醫療器械 / 의료 기기
	Authorized Representative in the European Community/European Union / Représentant autorisé établi au sein de la Communauté européenne/l'Union européenne / Bevollmächtigte Vertretung in der Europäischen Gemeinschaft/Europäischen Union / Rappresentante autorizzato nella Comunità europea/Unione europea / Representante autorizado en la Comunidad Europea/Unión Europea / 歐洲共同體/歐洲聯合的認定代理業者 / Representante Autorizado na Comunidade Europeia/União Europeia / Officiële vertegenwoordiger in de Europese Gemeenschap / Europese Unie / Auktoriserad representant inom Europeiska gemenskapen/Europeiska unionen / Autoriseret repræsentant i Det Europæiske Fællesskab/Den Europæiske Union / Valtuutettu edustaja Euroopan yhteisössä/Euroopan unionissa / Ευκρεμμένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα/Ευρωπαϊκή Ένωση / Upraważony przedstawiciel we Wspólnocie Europejskiej / Unii Europejskiej / Avrupa Topluluğu'nda/Avrupa Birliği'nde Yetkili Temsilci / Уполномоченный представитель в Европейском сообществе / Авторизованý зástupce v Evropském společenství/Evropské unii / Meghatalmazott képviselő az Európai Közösségekben/Európai Unióban / Oprávněný zástupca v Európskom spoločenstve/Európskej únii / Autorisert representant i Det europeiske felleskapet / EU / Pooblašćeni predstavnik v Evropski skupnosti/Evropski uniji / Упълномощен представител в Европейската общност/Европейския съюз / Reprezentant autorizat în Comunitatea Europeană/Uniunea Europeană / Volitatud esindaja Euroopa Ühenduses / Euroopa Liidus / Pilnvarotais päristävis Eiroopas Korienä/Eiropas Savienībā / Ovlašteni predstavnik u Europskoj zajednici / Europskoj uniji / Ovlašćeni predstavnik u Evropskoj zajednici/Evropskoj uniji / Įgaliotasis atstovas Europos Bendrijoje / Europos Sąjungoje / 欧洲共同体/欧洲联盟的授权代表 / 歐洲共同體/歐洲聯盟的授權代表 / 유럽 공동체/유럽 연합의 공인 대리인
	Caution / Attention / Vorsicht / Attenzione / Precaución / 注意 / Cuidado / Opgeet / Försiktighetsåtgärd / Forsigtig / Varoitus / Προσοχή / Prestroga / Dikkat / Внимание / Výstraha / Figyelem! / Upozornenie / Advarsel / Pozor / Внимание / Atenție / Ettevaatus / Uzmaniбу! / Oprez / Oprez / Dmesio / 注意 / 주의
	Unique Device Identifier / Identifiant unique du dispositif (UDI) / Eindeutige Geräteerkennung / ID univoco del dispositivo / Identificador único del dispositivo / 機器固有識別子 / Identificador Único de Dispositivo / Unieke apparaat-id / Unik enhetsidentifikere / Unik enheds-ID / Yksilöllinen laitetunniste / Μοναδικό αναγνωριστικό συσκευής / Niepowtarzalny identyfikator wyrobu / Benserzsid Cihaz Kimliği / Doğal kauçuk lateksten yapılmamıştır. / Уникальный идентификационный номер изделия / Jedinečný identifikátor zařízení / Egyedi eszközazonosító / Jedinečný identifikátor zariadenia / UDI (Unique Device Identifier) / Edinstven identifikator naprave / Унікален ідентифікатор на изделиї / Identificator unic al dispozitivului / Seadme kordumatu identifitseerimistunnus / Ierices unikālais identifikators / Jedinstveni identifikator uredaja / Jedinstveni identifikator uredaja / Unikalus prietaiso identifikatorius / 唯一设备标识符 / 唯一設備標識符 / 기기 고유 식별자
Not made with natural rubber latex. / Ne contient pas de latex de caoutchouc naturel. / Ohne Naturkautschuklatex hergestellt / Non fabbricato con lattice di gomma naturale. / No está fabricado con látex de caucho natural. / 天然ゴムラテックス製ではありません。 / Não fabricado com látex de borracha natural. / Niet vervaardigd met natuurlijk rubber latex. / Ej tillverkad med latex naturgummi. / Ikke fremstillet med naturgummilátex. / Ei sisällä luonnonkumilátexia. / Δεν κατασκευάζεται με τη χρήση φυσικού λάτεξ. / Nie jest wykonana z naturalnego lateksu gumowego. / При производстве не использовался латекс натурального каучука. / Při výrobě nebyl použit přírodní kaučukový latex. / Nem természetes gummilátexből készült. / Nie je vyrobené z prírodného latexu. / Ikke laget av naturlig gummilátex. / Ni izdelano iz naravnne gume iz lateksa. / Не съдържа естествен каучуков латекс. / Nu este fabricat din latex de cauciuc natural. / Pole valmistatud loodusliku kummilátexiga. / Nesatur dabīgā kaučuka lateksu. / Nije proizveden od prirodnog gumenog lateksa. / Ne sadrži prirodni gumeni lateks. / Pagaminta nenaudojant gamtinio kaučiuko lateksu. / 并非由天然胶乳制成。 /並非由天然膠乳製成。 / 천연 고무 라텍스 제조되지 않음.	



EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Summit Medical LLC
815 Vikings Parkway, Suite 100
St. Paul, MN 55121 | USA

P: 1-888-229-2875 | +1 651-789-3939
F: 1-888-229-1941 | +1 651-789-3979
www.summitmedicalusa.com