SYMBOL REFERENCE TABLE / TABLEAU DE RÉFÉRENCE DES SYMBOLES / TABELLE: SYMBOLLEGENDE / TABELA DE REFERÊNCIA DE SÍMBOLOS / TABLA DE REFERENCIA DE LOS SÍMBOLOS / LEGENDA DEI SIMBOLI	
	Consult instructions for use / Consulter le mode d'emploi / Gebrauchsanweisung beachten / Consultar as instruções de utilização / Consultar las instrucciones de uso / Consultare le istruzioni per l'uso
	Quantity / Quantité / Menge / Quantidade / Cantidad / Quantità
	For US audiences only / Réservé à la vente aux États-Unis / Nur für die USA bestimmt / Apenas para audiências dos EUA / Para usuarios de Estados Unidos / Solo per un pubblico statunitense
	Catalog number / Numéro de référence / Bestellnummer / Número de catálogo / Número de catálogo / Codice articolo
	Lot number / Numéro de lot / Chargennummer / Número do lote / Número de lote / Numero di lotto
	Use by date / Date limite d'utilisation / Verwendbar bis / Prazo de validade / Fecha de caducidad / Data di scadenza
	Sterilized using gamma radiation / Stérilisé aux rayons gamma / Mit Gammastrahlen sterilisiert / Esterilizado com radiação gama / Esterilizado con radiación gamma / Sterilizzato con raggi gamma
	Do not re-use / Ne pas réutiliser / Nicht wiederverwenden / Não reutilizar / No reutilizar / Non riutilizzare
	Do not use if package is damaged / Ne pas utiliser si l'emballage est endommagé / Beschädigte Verpackung nicht verwenden / Não utilizar se a embalagem estiver danificada / No utilizar si el envase está dañado / Non utilizzare in caso di imballaggio danneggiato
	Caution / Attention / Vorsicht / Atenção / Atención / Attenzione
	Manufactured by / Fabriqué par / Hergestellt von / Fabricado por / Fabricado por / Fabbriacato da
	Authorized Representative in the European Community / Représentant autorisé établi au sein de la Communauté européenne / Bevollmächtigte Vertretung in der Europäischen Gemeinschaft / Representante Autorizado na Comunidade Europeia / Representante autorizado en la Unión Europea / Rappresentante autorizzato nella Comunità europea
<i>Not made with natural rubber latex / Ne contient pas de latex de caoutchouc naturel / Ohne Naturkautschuklatex hergestellt / Não é fabricado com látex de borracha natural / No está fabricado con látex de caucho natural. / Non fabbricato con lattice di gomma naturale</i>	

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Rhino Rocket® Nasal Packs Instructions for Use

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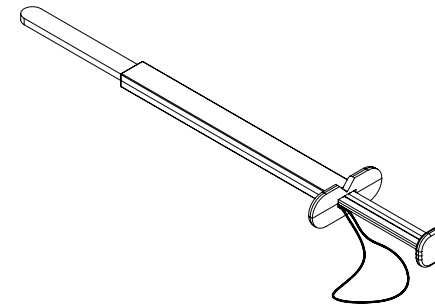
Rhino Rocket® Nasal Packs (with applicator)

Descriptions

- Summit Medical Nasal Packings are disposable, single-use devices.
- Rhino Rocket Nasal Packing is available in a variety of sizes. Each nasal packing catalog number is printed on the pouch label.
- Rhino Rocket Nasal Packings are pouched and provided sterile.

Do not re-sterilize.

Shelf Life

Use this device before the expiration date indicated on the label.

Intended Use

Absorption of postoperative drainage and epistaxis management.

Indication for Use

For use following intranasal surgery, surgical or non-surgical trauma, and/or epistaxis.

EN

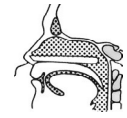
Figure 1

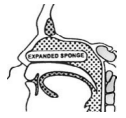
Figure 2
(Expanded Sponge)

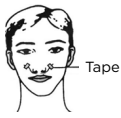
Figure 3

Instructions for Use

NOTE: Before opening, visually inspect the outer pouch and pouch seal for voids or openings.

DO NOT USE if the outer pouch or pouch seal is not intact.

1. Before Insertion: Verify the correct style and size for the desired procedure. Open the product only in a sterile field. Open by pulling and separating the pouch seal.

2. Insertion:

• **Rhino Rocket:** Using the applicator, insert the nasal pack along the floor of the nose until the entire nasal pack is within the nose (See Figure 1). Remove the applicator. The applicator is only used to insert the nasal pack.

3. After Insertion: The moisture sensitive sponge should begin expanding (See Figure 2). If necessary, hydrate the outer edge of the packing using saline or sterile water. Administer until the packing begins to expand. Secure the packing by placing the retention string around the ear AND taping the string to the side of the face (See Figure 3).

Caution: Tape string to face to prevent packing migration.

4. Before Removal: Rehydrate the packing using saline or sterile water. Administer until the packing is fully saturated. Allowing the packing to remain in place for 5 - 10 minutes after rehydration facilitates removal.

5. Removal: Use forceps to grasp the pack and gently withdraw.

6. Disposal: Properly dispose of device in accordance with biohazard regulations.

WARNINGS

- Sterile. Single-Use Only.
- Do not use if pouch is opened or damaged.
- Do not leave packing in place longer than 24 hours.
- Removal must be performed by a physician.
- Always rehydrate before removal.
- If packing string is not taped to cheek, or if packing is dislodged, respiratory problems can occur.

TOXIC SHOCK SYNDROME (TSS)

In rare instances, the physiochemical conditions associated with nasal surgery, both with and without nasal packings, may present a risk of toxic shock syndrome (TSS). Warning signs of TSS include: sudden fever (usually 102°F or more), vomiting, diarrhea, dizziness, fainting (or near fainting when standing up), and/or a rash that looks like sunburn. If these or other signs of TSS appear, remove the packing at once and seek medical attention immediately. Rhino Rocket Nasal Packs exhibit no antimicrobial properties, they are not bacteriostatic towards pre-existing infections, nor do they prevent the occurrence of new infections. In the case of pre-existing infections, it is recommended that appropriate treatment be instituted.

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All Rights Reserved © 2021 Summit Medical, an Innovia Medical Company.