



MicroEdge™ Myringotomy Blades

Lames de bistouri pour
myringotomie


Myringotomiemesser

Cuchillas para miringotomía

Bisturi per miringotomia

Myringotomiesjes

Myringotomi-knivblad

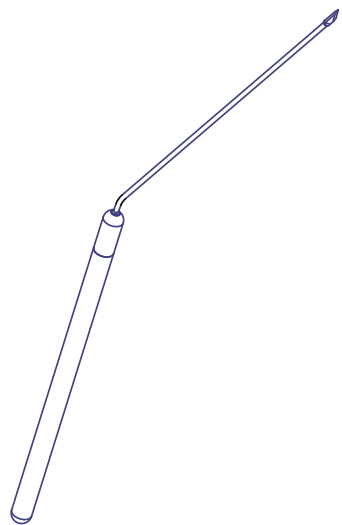

Summit Medical Model BL-0425
Blade Handle, Collet Style, Non-Sterile
Porte-lame à collet pour le modèle
BL-0425 de Summit Medical, Non-Stérile
Summit Medical Modell BL-0425
Messergriff, Spannhülse, Nicht Steril
Mango para cuchilla, tipo Pinza
Modelo BL-0425 de Summit Medical,
no estéril
Modello BL-0425 Summit Medical
Manico per bisturi con bussola di
chiusura, non sterile
Summit Medical model BL-0425
Meshandvat, met kraag, Niet-Steriel
Summit Medical modell BL-0425
Knivskaft, klemmetype, Ikke Steril

FIG. 1 | ABB. 1 | AFB. 1

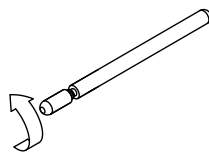


FIG. 2 | ABB. 2 | AFB. 2

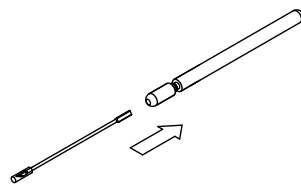


FIG. 3 | ABB. 3 | AFB. 3

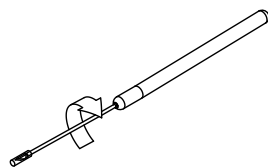


FIG. 4 | ABB. 4 | AFB. 4


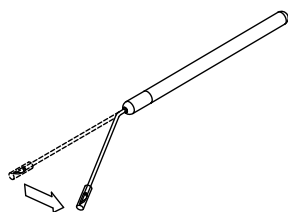


FIG. 5 | ABB. 5 | AFB. 5

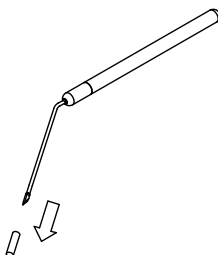

Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning

Summit Medical Model BL-1425
Blade Handle, Adaptor Style, Non-Sterile
Porte-lame adaptateur pour le modèle
BL-1425 de Summit Medical, Non-Stérile
Summit Medical Modell BL-1425
Messergriff, Adapter Nicht Steril
Mango para cuchilla, tipo Adaptador
Modelo BL-1425 de Summit Medical,
no estéril
Modello BL-1425 Summit Medical
Manico per bisturi con adattatore,
Non Sterile
Summit Medical model BL-1425
Meshandvat, met adapter, Niet-Steriel
Summit Medical modell BL-1425
Knivskaft, adaptertype, Ikke Steril

FIG. 6 | ABB. 6 | AFB. 6

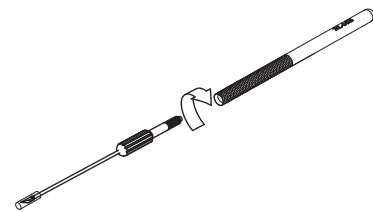


FIG. 7 | ABB. 7 | AFB. 7



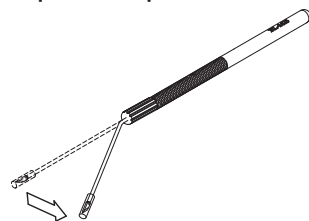


FIG. 8 | ABB. 8 | AFB. 8

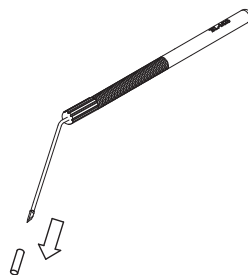

Popular Styles of Flat Blade Handles
Modèles courants de porte-lames
plats pour bistouris
Gängige Ausführungen für flache
Messergriffe
Tipos populares de mangos para
cuchilla plana
Manici per bisturi a lama piatta
Populaire stijlen van platte
meshandvatten.
Skaft for populære typer av flate
knivblad

FIG. 9 | ABB. 9 | AFB. 9

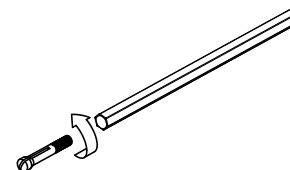

MicroEdge™ Myringotomy Blades and Handles

Before using, read the following
information:

INDICATIONS FOR USE

Where an incision in the tympanic membrane is
required.

CONTRAINDICATIONS

None known.

PRECAUTIONS

The tip protector should remain in place until blade
is ready to use to prevent damaged to the cutting
edge or injury.

POSSIBLE ADVERSE REACTIONS

Minor scar tissue from incision.

Infection due to airborne or water contaminants.

INSTRUCTIONS FOR USE

1. ASSEMBLY

- A. Model BL-0425 Blade Handle, Collet Style.
1. Check to ensure that the handle and blade
are compatible (See Chart Below).
 2. Remove blade from packaging using
sterile technique.
 3. Loosen (but do not remove) locking nut,
FIG. 1
 4. Insert the blade into handle. **FIG. 2**
 5. Tighten locking nut securely to retain
blade. **FIG. 3**
 6. Form blade to desired angle, **FIG. 4**
 7. Remove tip protector from end of blade
and the Myringotomy Incision System is
ready for use. **FIG. 5**

The Summit Medical Model BL-0425 Blade
Handle, Collet Style is intended for use with the
following models of myringotomy blades:

Model BL-0150	Model BL-0160
Model BL-0151	Model BL-0170
Model BL-0152	

B. Model BL-1425 Blade Handle, Adaptor Style.

1. Check to ensure that the handle and blade
are compatible (See Chart Below).
2. Remove blade from packaging using
sterile technique.
3. Insert the threaded end of the adapter into
handle and tighten firmly. **FIG. 6**
4. Form blade to desired angle, **FIG. 7**
5. Remove tip protector from end of blade
and the Myringotomy Incision System is
ready for use. **FIG. 8**

The Summit Medical Model BL-1425 Blade
Handle, Adaptor Style is intended for use with
the following models of myringotomy blades:

Model BL-1150	Model BL-1160
Model BL-1151	Model BL-1170
Model BL-1152	

The Summit Medical Blade with Adapter is also
compatible with other leading brands of Flat
Blade Handles.

C. Popular Flat Blade Handles

1. Remove the existing blade collet from the
handle and discard. **FIG. 9**
2. Complete blade with adaptor installation
as shown in Step 3,4 and 5 of Section 1B.

2. CLEANING (HANDLE ONLY)

Remove blade from handle before cleaning.
The handles should be cleaned by scrubbing
with a non-oily soap and soft brush.
DO NOT use substances containing chlorides.

3. STERILIZATION (HANDLE ONLY)

It is recommended that the handle be sterilized
by steam autoclave.



Single-use, one-patient device will degrade if
reprocessed. No effective cleaning process has
been developed to prevent cross contamination.
Contamination of a reprocessed device may lead
to injury, illness or death of the patient.



Symbol Reference Key
Définition des symboles
Erklärung der Symbole
Definición de los símbolos
Legenda dei simboli
Verklaring van symbolen
Nøkkel til symbolreferanser

	Consult instructions for use Consulter les instructions d'utilisation Gebrauchsanweisung beachten Consulte las instrucciones de uso Consultare le istruzioni per l'uso Raadpleeg de instructies voor gebruik Consult instruksjonene for å bruke	
	Quantity Quantité Menge Cantidad	Quantità Aantal Menge
	For US audiences only Réservé à la vente aux États-Unis Nur für die USA bestimmt Solo para usuarios de los EE. UU. Solo per pubblico statunitense Alleen voor gebruik in de VS Bare for brukere i USA	
	Catalog Number Numéro de référence Katalognummer Número de catálogo Numero di catalogo Catalogusnummer Katalognummer	
	Lot Number Numéro de lot Chargenbezeichnung Número de lote Numero di lotto Partijnummer Partinummer	
	Use By Utiliser avant Verwendbar bis Fecha de caducidad Data di scadenza Uiterste gebruiksdatum Må brukes før	
	Manufactured Date Date de fabrication Herstellungsdatum Fecha de fabricación Data di produzione Produsert dato Fabricagedatum	
	Sterilized by Ethylene Oxide Stérilisé à l'oxyde d'éthylène Mit Ethylenoxid sterilisiert Esterilizado mediante óxido de etileno Sterilizzato con ossido di etilene Gesteriliseerd met ethyleenoxide Sterilisert med etylenoksid	BL-0150 BL-1150 BL-0151 BL-1151 BL-0152 BL-1152 BL-0160 BL-1160 BL-0170 BL-1170
	Manufactured By / Fabriqué par/ Hergestellt von/ Fabricado por/ Nome del produttore / Geproduceerd door / Produsent	
	Do not use if package is damaged / Ne pas utiliser si l'emballage est endommagé / Bei beschädigter Verpackung nicht verwenden / No utilizar si el envase está dañado / Non utilizzare in caso di imballaggio danneggiato / Niet gebruiken indien verpakking beschadigd is / Må ikke brukes hvis pakningen er skadet	



Summit Medical LLC
815 Vikings Parkway, Suite 100
St. Paul, MN 55121 | USA

P: 1-888-229-2875 | +1 651-789-3939
F: 1-888-229-1941 | +1 651-789-3979
www.summitmedicalusa.com

