

SPEEDI-WICK® Ear Pack

Instructions for Use

COMPOSITION

The Speedi-Wick® Ear Pack is composed of Expandacell® polyvinyl alcohol foam.

INDICATIONS

For use in the treatment of otitis externa.

INSERTION

Grasp Speedi-Wick® with forceps and place into the inflamed ear canal.

AFTER INSERTION

Hydrate the wick with an aqueous otic suspension. The suspension will cause the wick to expand and conform to the ear canal while allowing the medication to remain in contact with the canal lining.

TO REMOVE

Grasp Speedi-Wick® with forceps and gently remove. Discard the Speedi-Wick® in appropriate trash receptacle.



CAUTION

- Remove pack within four days. A new pack may be inserted.
- Always hydrate the pack with an aqueous otic suspension.



SINGLE USE ONLY









Single-use, one-patient device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of the patient.



TOXIC SHOCK WARNING

Warning signs of Toxic Shock Syndrome (TSS) include sudden fever, usually 102° F or higher, and diarrhea, vomiting, fainting or near fainting when standing up, dizziness, or a rash that resembles sun burn. Physicians are encouraged to alert all patients to be aware of these symptoms and to report any such symptoms immediately to the physician. In the case of pre-existing infections, it is recommended that anti-microbial treatment be instituted before use of this packing.

SYMBOL REFERENCE KEY

	Consult instructions for use
	Quantity
REF	Catalogue number
LOT	Lot number/Batch code
	Use-by-date
STERILE R	Sterilized using irradiation
	Do not reuse
	Do not use if package is damaged and consult instructions for use
	Caution
	Manufacturer
	Distributor
R_x Only	Caution: US Federal law restricts this device to sale by or on the order of a physician



ANTHONY PRODUCTS, INC.
7740 Records Street
Indianapolis, IN 46226

 **AN
INNOVIA
MEDICAL
COMPANY**



Summit Medical LLC
815 Vikings Parkway, Suite 100
St. Paul, MN 55121 | USA

P: 1-888-229-2875 | +1 651-789-3939
F: 1-888-229-1941 | +1 651-789-3979
www.summitmedicalusa.com