




DTR Medical®

AN
INNOVIA
MEDICAL
COMPANY

Sterile Single-use
The beauty is in the evolution

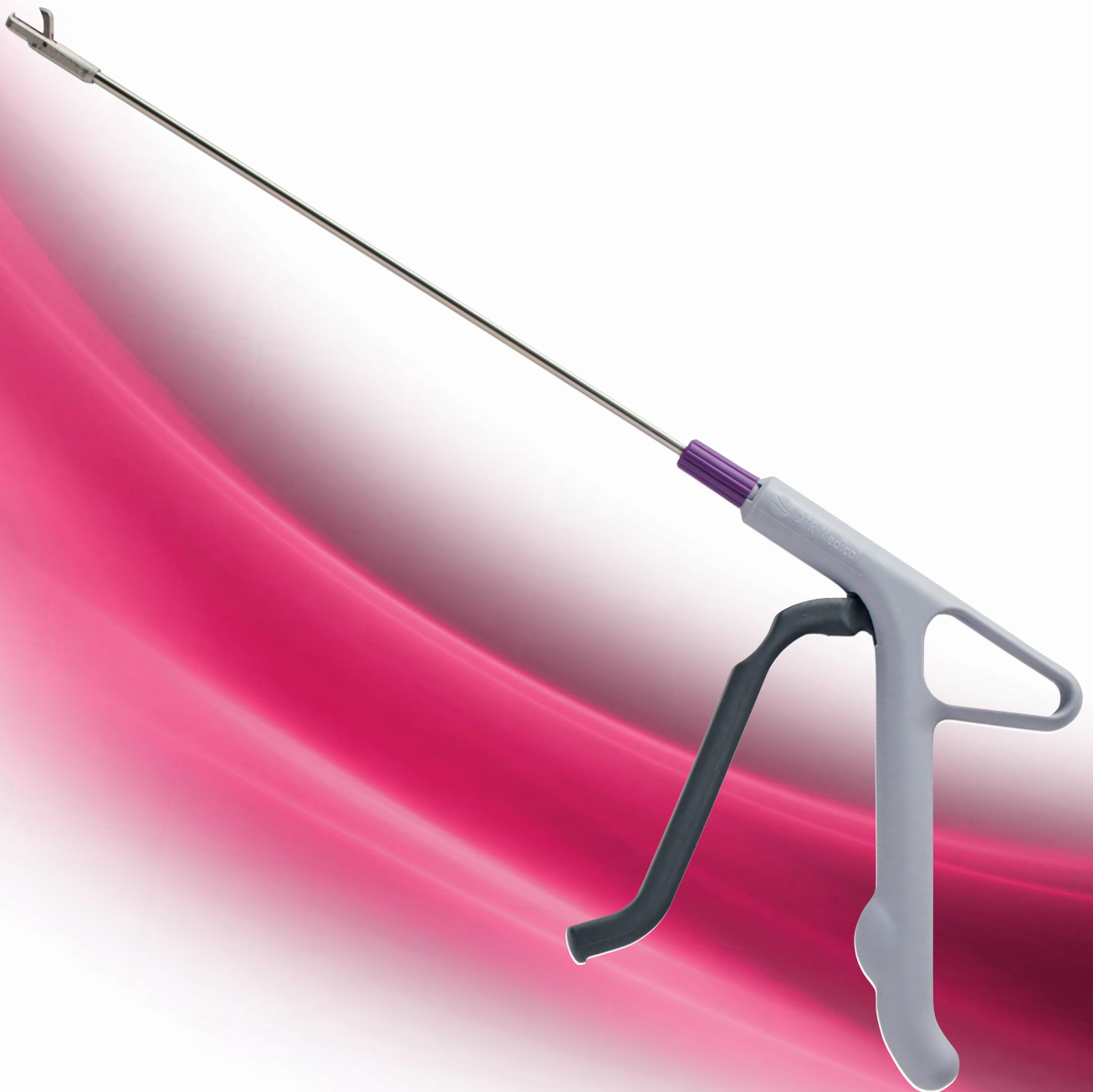



About Innovia Medical

Innovia Medical is a family of specialty surgical companies that are passionately committed to partnering with medical professionals to help elevate the delivery of patient care and improve clinical outcomes.

ELEVATING CARE. IMPROVING OUTCOMES.

Innovia Medical companies develop and manufacture a portfolio of product solutions designed with both the medical professional and their patients in mind, to deliver care that:

- Saves time for both the medical professional and patient
- Delivers a consistent and predictable care experience
- Improves patient confidence with reduced recovery times and less discomfort

Innovia Medical does this through a commitment to innovation leadership, unsurpassed product quality and a dedication to provide responsive, personalized customer service.

INNOVATION LEADERSHIP.

Innovation leadership is fueled through our dedication to collaboration with medical professionals. We regularly seek feedback and input from physicians, nurses, and other medical professionals through product research, testing and development.

This allows us to truly understand the solutions medical professionals need, and guides our innovative culture focused on inventing new, industry-leading solutions and reinventing current solutions that make ordinary products extraordinary, across the specialty surgery spectrum.

UNSURPASSED PRODUCT QUALITY.

Our products deliver proven solutions and dependability which is the result of rigorous quality control and a relentless commitment to ongoing manufacturing improvements. We use only the highest quality materials and subject all products to rigorous inspection protocols. This is why we're proud to have so many customers who have continued to rely on our solutions for more than forty years.

RESPONSIVE, PERSONALIZED CUSTOMER SERVICE.

From our sales representatives to our customer service team, every associate at Innovia Medical strives to deliver an unrivaled customer experience. This begins by ensuring that each customer receives the product that is right for their specific needs. We provide assistance through technical support and live customer service representatives with extensive product knowledge, available throughout the day. We deliver quick and thorough responses, every time—from order and shipping confirmations to delivery date communications and more ensuring continuous satisfaction.

We know you're committed to providing the best patient care experience possible and that's why we commit to being your trusted partner helping you every step of the way. At Innovia Medical, we are not just products, we are part of the patient care solution.

Learn more at www.innoviamedical.com

Contents

Aspirating Needles	4
Spackman Insufflation Cannula	5
Cervical Rotating Biopsy Punch	6-7
Tischler Biopsy Punches	8
Endometrial Sampling	9
LLETZlearn® Training Simulator	10-11
Desjardins Forceps	12
Forceps <i>Including Rampley Sponge Holder, Teale Vulsellum, Rotunda Vulsellum & Bonney Uterine Polyp Forceps</i>	13-14
Scissors <i>Including Episiotomy, Stevens Tenotomy, American Pattern Umbilical Cord, Busch Umbilical, Mayo, Metzenbaum, Supercut Baby Metzenbaum</i>	15-17
LLETZ Loops & Balls	18
Dental Syringes	19
Endocervical Specula <i>Including Kogan & Kurihara Endocervical Speculum & Sims Uterine Sound</i>	19
Metal Cusco Specula <i>Including Insulated with Smoke Tube, Non-Insulated with Smoke Tube & Non-Insulated without Smoke Tube</i>	20-21
Electrosurgical Pencils	22
Scope Warmer	22
Instrument Wipes	22

Cannula and Biopsy Needles


ASPIRATING NEEDLES

ASN2018	Aspirating Needle 335 x 3.5mm (18g) with Luer	20 units
ASN5018	Aspirating Needle 335 x 3.5mm (18g) with Luer	50 units

- Stainless steel cannula with 18g needle for laparoscopic aspiration
- Single-use avoids difficult to clean cannula and needle with stepped joint
- Extra-long 335mm length suitable for all patients of any BMI
- Optimum first-time sharpness of needle for ease of use
- Available in 3.5 diameter cannulas complete with Luer connection

“Accurate during procedure,
effective with good results.”

Mid Cheshire NHS Trust

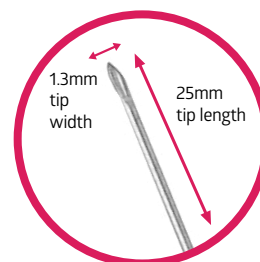



ASN2018



Spackman Insufflation Cannula

- Stainless Steel
- Single-use removes difficulty of cleaning occluded acorn end
- Improved adjustable Vulsellum plate with stronger hooks to aid placement
- Specially designed for an effective manipulation of the Uterus and injection of fluid
- Atraumatic tips to reduce uterine perforation risk
- Improved overall quality

Spackman Insufflation Cannula

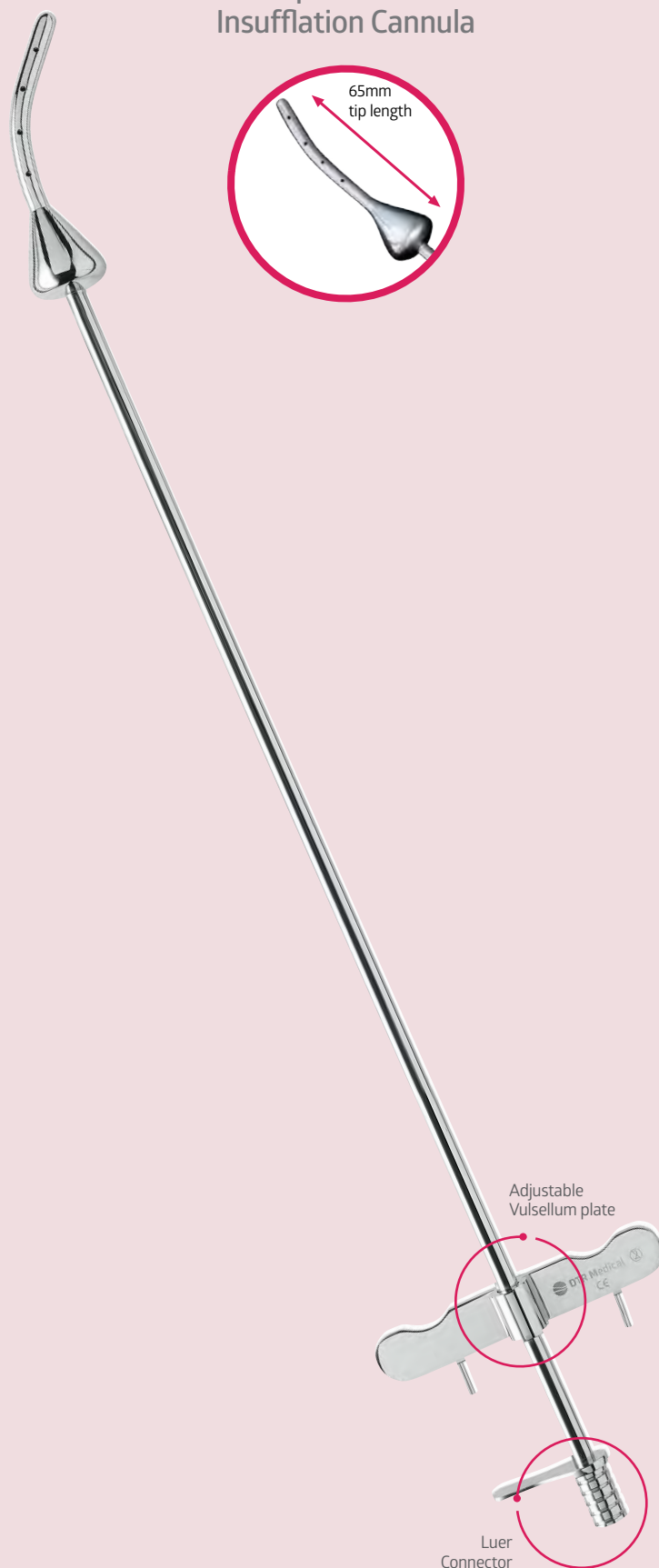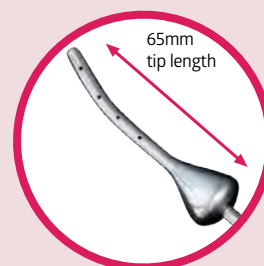

Looking for Vulsellum Forceps?
See p13

SPACKMAN INSUFFLATION CANNULA


SPCS1001 Spackman Insufflation Cannula 10 units

Cervical Rotating Biopsy Punch

- Latest generation top jaw now stronger than Titanium, and 4x the strength of previous jaw
- Allows a Biopsy to be taken from a Hardened Cervix
- Low profile jaw provides better access to the transformation zone when migrating into the Endocervical Canal
- 360° rotation enables enhanced positioning and patient interaction
- Ensures a sharp, clean cut reducing Cervical trauma and patient discomfort
- Ergonomic handle designed for comfort
- Provides value-based benefits, improving procedure quality, patient safety, and overall cost savings
- Lightweight compared to alternatives
- Ideal for busy Colposcopy Clinics, with high-quality samples taken first-time
- Useful for increased HPV testing demands
- Single-use eliminates cross-contamination, inadequate samples, and repeat procedures



Read our "Importance of a Good Biopsy in Reducing Time to Accurate Diagnosis" Infographic

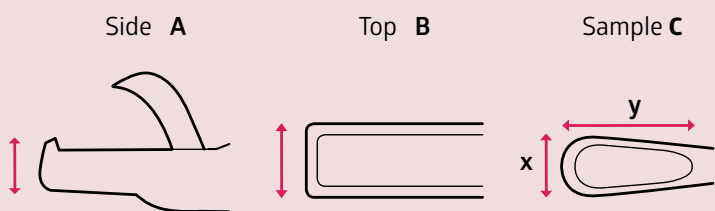

"It's excellent that you have improved the new jaw on the Cervical Rotating Biopsy Punch, it is much stronger and removes previous issues we had taking a biopsy from a hardened cervix. It is great that you have taken our feedback on board and made this improvement to the instrument, thank you."

*Helen Crookes,
Senior Sister, Calderdale Royal Hospital*

"I love the Cervical Rotating Biopsy Punch. You can really notice the difference in how the product performs so much, that I do not have any problems using it on more demanding patients. The product feels sharp like a hot knife through butter."


*Gill Costello,
Sister, Northwick Park Hospital*

"The new forceps are really decent. No problems at all and very good specimen is obtained."

*Garima Srivastava,
Homerton University Hospital*


BENEFITS OF A LOW PROFILE JAW

	A	B	Sample	Rotation
Tischler	3.6mm	5.2mm	x 3.1mm y 7.5mm	X
Cervical Rotating Biopsy Punch	3.6mm	5.2mm	x 3.0mm y 7.5mm	✓
Baby Tischler	3.8mm	5.2mm	x 2.6mm y 8.5mm	X

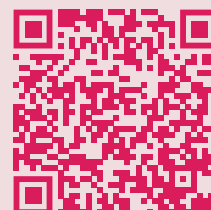

POINT CAMERA TO
VIEW INFOGRAPHIC

Cervical Rotating Biopsy Punch

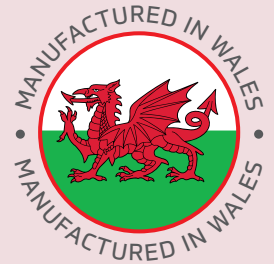

MediWales

**Innovation
Award Winners**

Low profile jaw
enables easier access

Rotating head provides
easier positioning and
secures visibility

Ergonomic handle
designed for comfort

CERVICAL ROTATING BIOPSY PUNCH

CBR1041 Cervical Rotating Biopsy Punch 10 units

Tischler Biopsy Punches

TISCHLER CERVICAL BIOPSY PUNCHES

CBT1001	Tischler Cervical Biopsy Punch 230mm	10 units
CBT1001B	Baby Tischler Cervical Biopsy Punch 230mm	10 units

- High quality stainless steel instruments for a consistent and precise biopsy
- Sharp first time action to minimise trauma



Endometrial Sampling

MEDGYN PRODUCTS

022730	Medgyn Pipette IV hole	25 units
022720	Medgyn Endosampler	25 units

Pipette


- Four-Hole Pipette provides improved sampling
- Pipette has enhanced flexibility and longitudinal strength to aid a quick and easy insertion
- Pipette is the instrument of choice when used to obtain clearly differentiated endometrial tissue without the need for anaesthesia
- Reliable supply, supported by strong service

Endosampler

- Designed to facilitate sampling for Gynaecological procedures such as Cytologic and Histologic diagnosis
- Enables Endometrial scraping and aspiration
- Conveniently bent with a sharp slot on the side
- Semi-ridged 3mm curved Curette and 10CC Syringe
- Features luer lock spring mechanism for larger and multiple sampling
- Engineered to prevent any potential backflow of the collected biopsy



Pipette IV

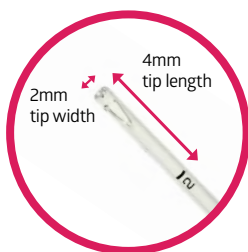

Endosampler



LLETZlearn® Training Simulator

- Unique Training Simulator for practical training
- Can be used with electro-surgical tools to imitate real life treatment
- Features an intuitive assembly that is easy to clean
- Available to use with any type of Return Electrode Pad
- Extremely beneficial as a training tool when used with our Cervical Rotating Biopsy Punch to practice taking a high-quality sample; as well as Colposcopy Coloured Loops and Ball electrodes for LLETZ procedures
- Awarded the MediWales "Partnership with the NHS Award"



Simple assembly.
Easy to clean.
Securely hold
specimen in place
during procedure.



REUSABLE


LLETZLEARN® TRAINING SIMULATOR

LTZ011 LLETZlearn® Training Simulator 1 unit



Preparation for the Procedure

- 1 **Insert** and secure a treatment speculum into the vagina model opening
- 2 **Attach** a suction tube to the smoke outlet pipe on the speculum
- 3 **Set-up** and align a Colposcope to obtain an optimum view of the sample
- 4 **Connect** a patient return electrode pad onto the stainless metal plate and attach the long cable to an Electrosurgical Machine (See Appendix 1)
- 5 **Attach** a pencil cable connector to the Electrosurgical Machine
- 6 **Assemble** the sterile electrically charged diathermy Loop to the head of the hand-held Pencil, using a small, medium or large loop. *Hold this instrument as you would a writing pencil ensuring it remains sterile. Do not touch the loop*
- 7 **Switch** on the power to the Electrosurgical Machine, to send power to the diathermy Loop and electrode Pad. If the contact between the electrode pad and the specimen is correct the indicator light on the Electrosurgical Machine will switch to **Green**
- 8 **Switch** on the power to the Colposcope. Focus the lens as required
- 9 **Set the power output** on the Electrosurgical Machine to the following settings;
 - Medium loop Cut: 50 – 60
 - Coagulation: 50 – 60 for specimen.
 - NOTE:** These settings are *ONLY* to be used when working on a specimen
- 10 **Switch** on the smoke evacuation suction system

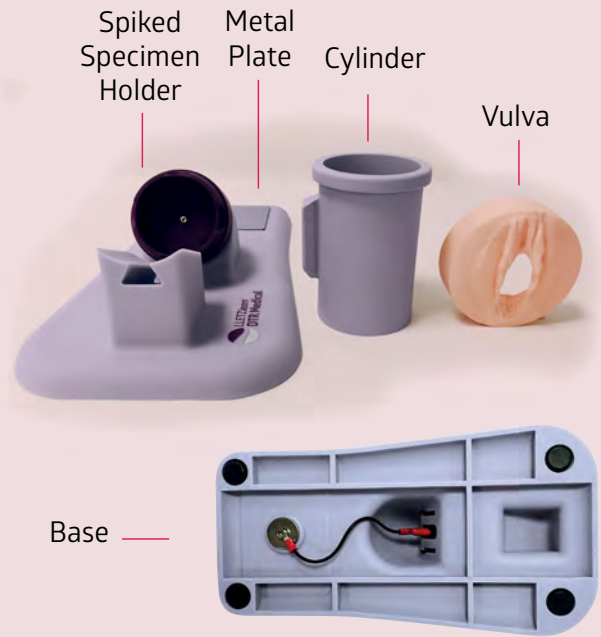

Aftercare

- 1 Turn off external power supplies
- 2 Remove all attached equipment used in steps 5-10
- 3 Unattach the cable connector
- 4 Detach the cylinder and clean down specimen holder with an alcohol wipe

Appendix 1.

How to attach the Patient Return Electrode Pad onto the stainless metal plate – Adult or Paediatric Type

Paediatric type: The advantage of using a paediatric pad is that it is smaller, fits easily onto the plate – and does not need to be folded.

Adult type:

- 1 Peel back the white paper backing 2/3 to expose the sticky surface (Image 1)
- 2 Fold the sticky pad ensuring the pad covers the blue safety strip and extends beyond the blue strip and press it down in order that it is folded flat (Image 2)
- 3 Remove the white paper from the remaining 1/3 of the pad and adhere this section to the stainless metal plate on the simulator
- 4 Attach the long cable to an Electrosurgical Machine



Desjardins Forceps


- Curved shaft for clearer visibility in the cervical os
- Smooth atraumatic edges for increased patient comfort
- Designed to meet specified Gynaecological requirements
- Used in endometrial polyp removal
- Ergonomic streamlined design improves visibility when removing tissue or opening the cervical os

“The Desjardins Forceps are very light and easy to use and will be more comfortable for patients. Have the advantage of having a curve on them and this means that you can actually target what you want without your hand or the stalk of the forceps being in your field of view. You actually can see into the endocervix much more easily than some of the larger forceps.”

*Dr Theodora Pepera,
St George's Hospital NHS Trust
& Parkside Hospital*

DESJARDINS FORCEPS

DJF2201 Desjardins Forceps 220mm 20 units

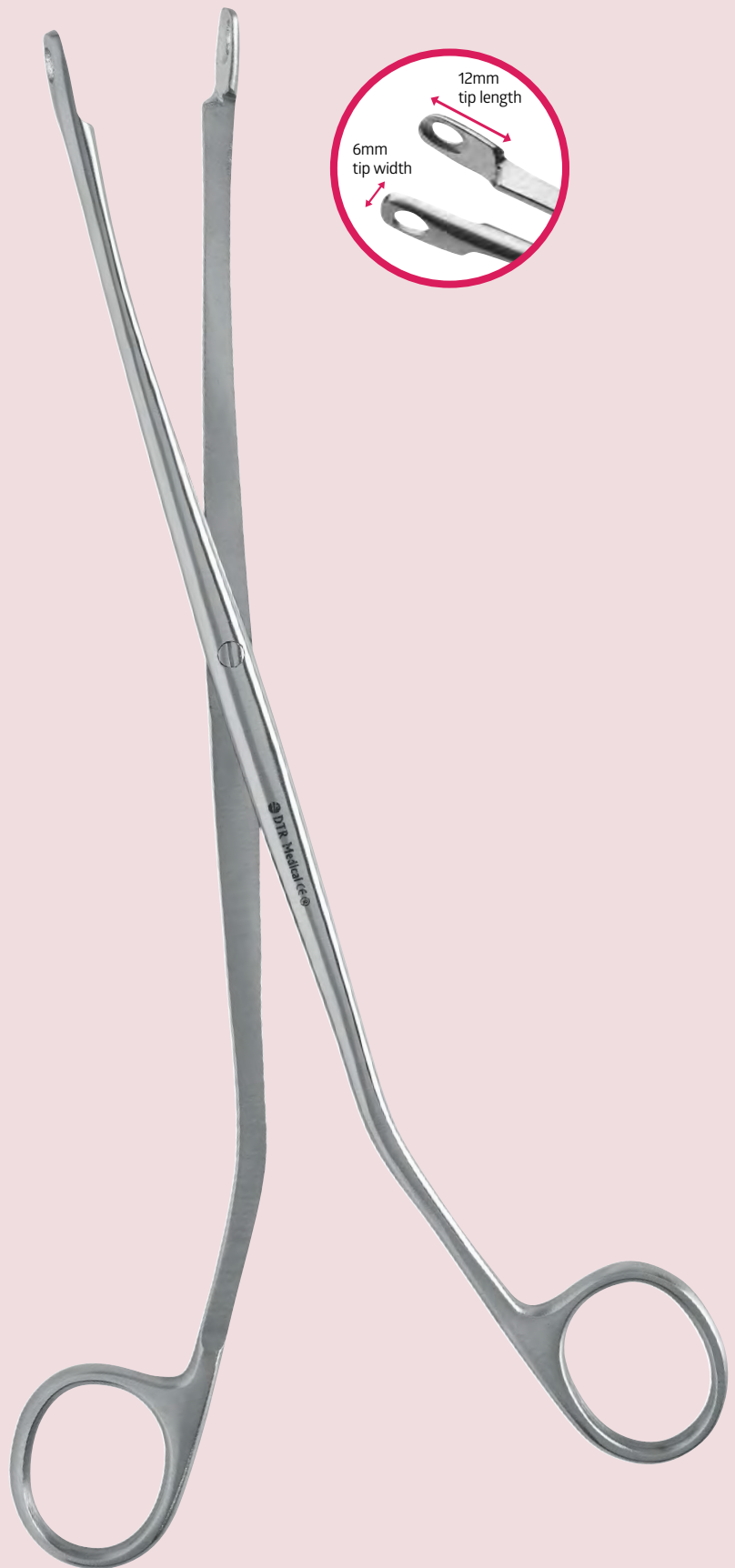

Forceps

RAMPLEY SPONGE HOLDERS

RSH2181	Rampley Sponge Holder 180mm	20 units
RSH2241	Rampley Sponge Holder 240mm	20 units

- Strong jaws with an effective locking ratchet for effectively holding swabs and gripping tissue during procedures
- Robust grip




Vulsellum Forceps

TEALE VULSELLUM FORCEPS

VFT1001	Teale Vulsellum Forceps	10 units
---------	-------------------------	----------


- High quality stainless steel for good tissue grasping
- Jaw with 3:4 teeth and 235mm length
- Curved to aid positioning
- Complementary to Spackman Insufflation Cannula

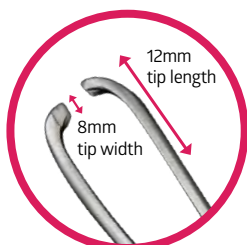

ROTUNDA VULSELLUM FORCEPS

VFR1001	Rotunda Vulsellum Forceps	10 units
---------	---------------------------	----------


- High quality stainless steel for good tissue grasping
- Jaw with 1:1 teeth and precise action
- Straight 254mm length to suit most procedures
- Complementary to Spackman Insufflation Cannula

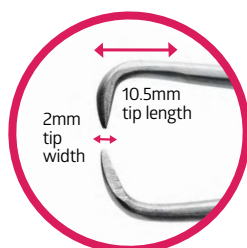

Looking for Spackman Insufflation Cannula?
See p5

Bonney Uterine Polyp Forceps

- Slender strong jaw with precision aligned tips, suitable for many procedures such as polyp and coil removal
- Effective ratchet that enables a strong and robust grip, superior to the plastic Rampley Sponge Holder alternatives
- 6mm head sizes that improve visibility and access

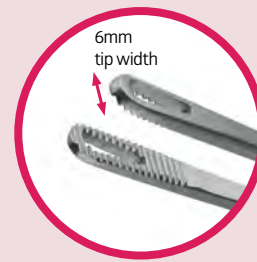


“Doctors are very happy with the Bonney Uterine Polyp Forceps and cannot see the difference with a reusable alternative.”

*St Peter's Hospital, Chertsey
Gynaecology Department*

“Strong with dependability and a narrow head for perfect endosurgical use.”

*New Victoria Infirmary, Glasgow
Gynaecology Department*

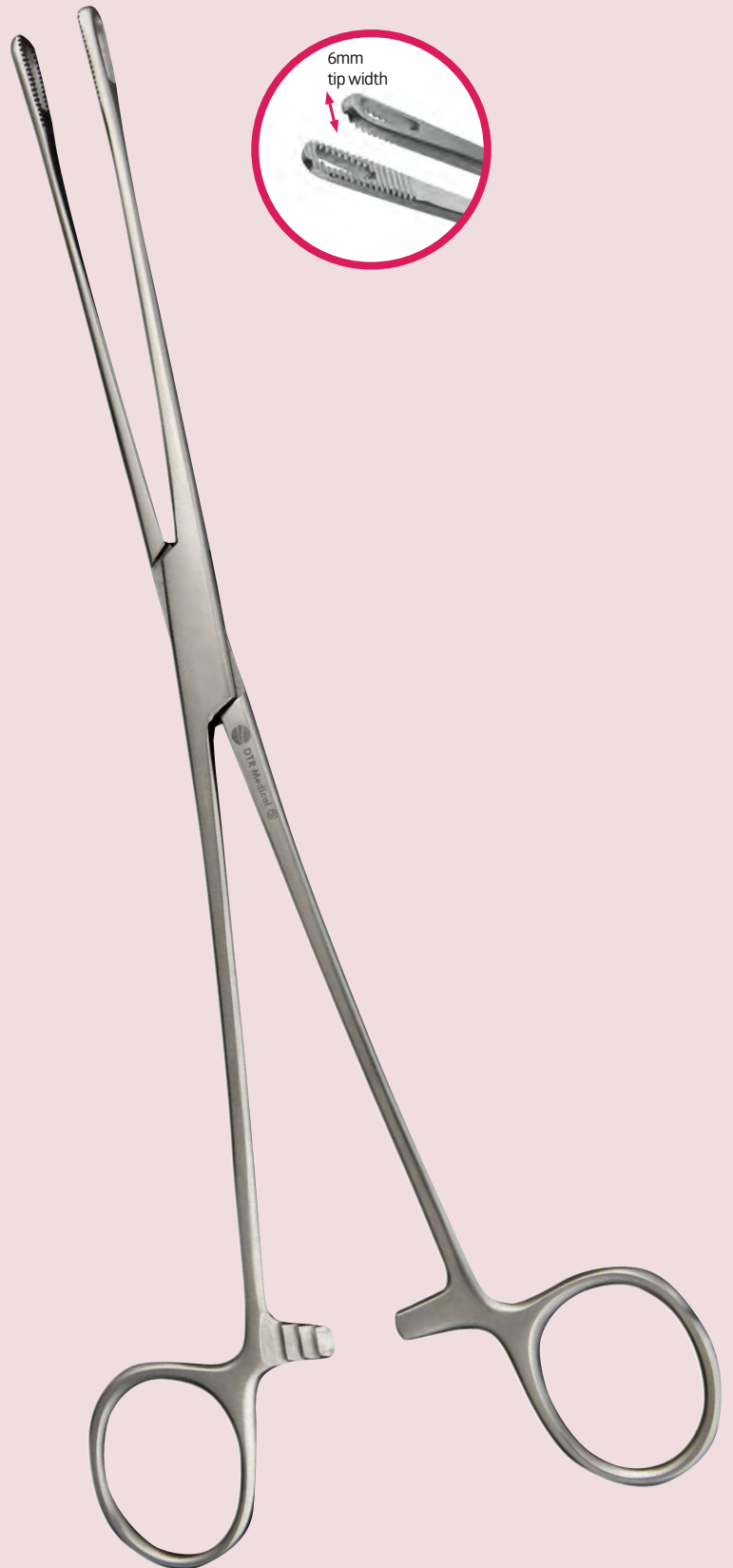

BONNEY UTERINE POLYP FORCEPS


BPF2001 Bonney Uterine Polyp Forceps 240mm 20 units

Scissors


EPISIOTOMY SCISSORS

WPS2018	Waldmann Perineum Episiotomy Scissors	20 units
BSS2001	Braun Stadler Episiotomy Scissors Angular 145mm	20 units

- High quality cut
- It has a slender sharp double-bevelled blade for precise cutting
- Shorted flat blunt blade for support
- Sharp blades facilitate insertion into the vagina

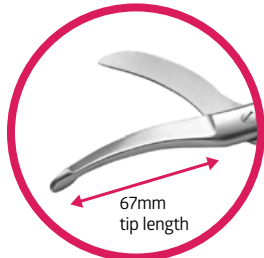

WPS2018

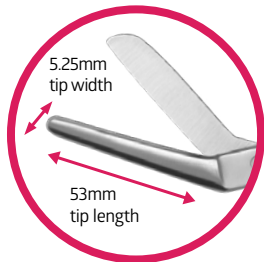

BSS2001



Waldmann Perineum Episiotomy Scissors




Braun Stadler Episiotomy Scissors


STEVENS TENOTOMY SCISSORS

TSS2001-S	Stevens Tenotomy Scissors 115mm Straight	20 units
TSC2001-S	Stevens Tenotomy Scissors 115mm Curved	20 units

- Stainless steel sharp/sharp scissors
- First time sharpness which cut even at the tips
- Smooth and elegant long handle
- Allows the surgeon increased grip and control when cutting in small or narrow areas

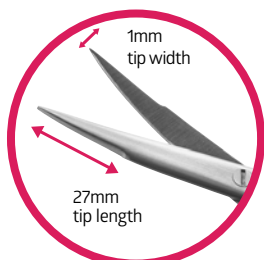

Curved

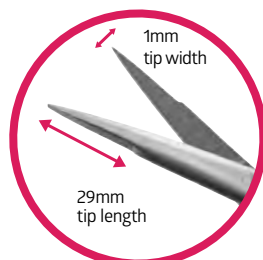

Straight



Stevens Tenotomy Scissors Straight

Scissors

NEW

SUPERCUT STEVENS TENOTOMY SCISSORS		
STSS2001	SuperCut Stevens Tenotomy Scissors 115mm Straight	20 units
STSC2001	SuperCut Stevens Tenotomy Scissors 115mm Curved	20 units


- Ideal for delicate cutting and dissecting soft tissue
- One serrated blade and one sharp cutting blade allows a superior grip and cut
- Available with straight or curved blades




IMAGE COMING SOON

SCISSORS		
USC2001	American Pattern Umbilical Cord Scissors 100mm	20 units
BUS2002	Busch Umbilical Scissors 160mm	20 units
MSS2141	Mayo Scissors Straight Bevelled Blade 140mm	20 units
MSC2141	Mayo Scissors Curved Bevelled Blade 140mm	20 units
MSS2171	Mayo Scissors Straight Bevelled Blade 170mm	20 units
MSC2171	Mayo Scissors Curved Bevelled Blade 170mm	20 units


- Wide range for different procedures
- Straight and curved variants
- Mayo Scissors provide the choice of curved or straight bevelled blades

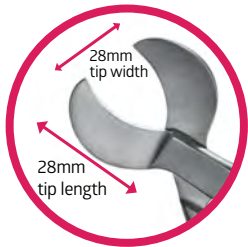

American Pattern Umbilical Cord Scissors

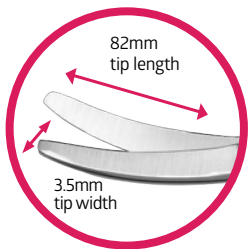


Busch Umbilical Scissors

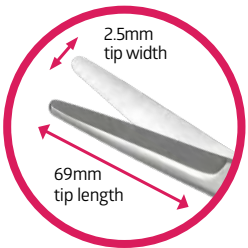


Straight

140mm

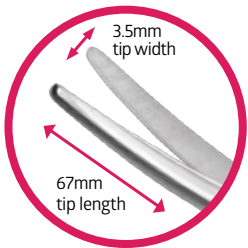

Curved

140mm


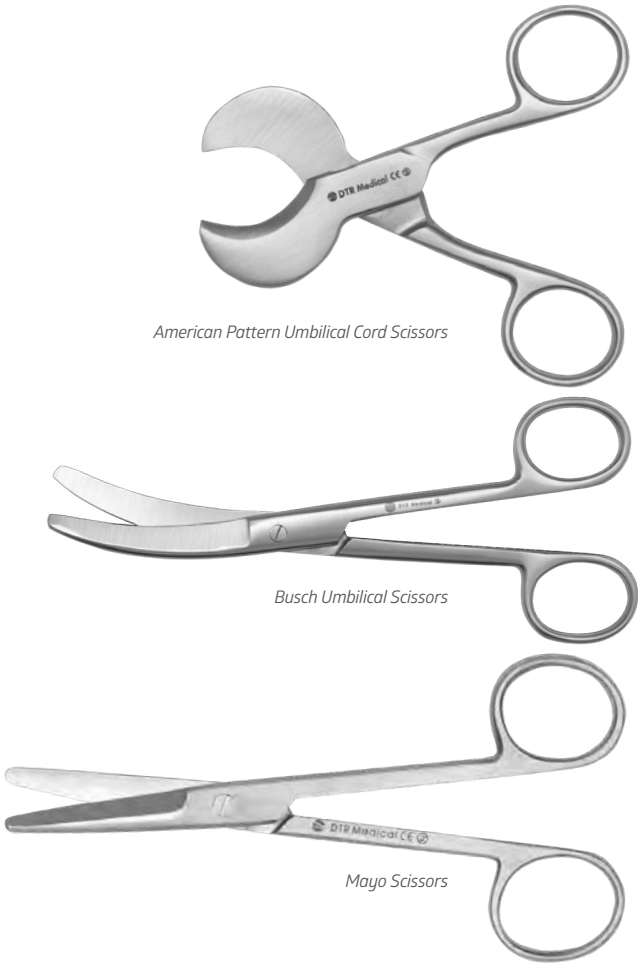


Scissors



METZENBAUM SCISSORS

ZSC2121-D	Baby Metzenbaum Scissors Curved 120mm	20 units
ZSC2141-D	Metzenbaum Scissors Curved 145mm	20 units
ZSC2161-D	Metzenbaum Scissors Curved 160mm	20 units
ZSS2181-D	Metzenbaum Scissors Straight 180mm	20 units
ZSC2181-D	Metzenbaum Scissors Curved 180mm	20 units
ZSS2201-D	Metzenbaum Scissors Straight 200mm	20 units
ZSC2201-D	Metzenbaum Scissors Curved 200mm	20 units



- First time performance for cutting delicate tissue and blunt dissection
- Specially engineered to be fine and precise

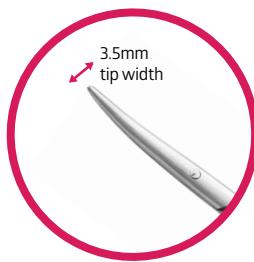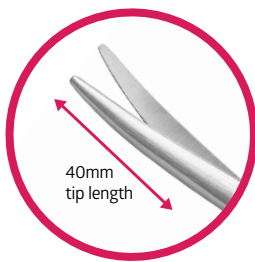

Baby Metzenbaum Scissors Curved 120mm

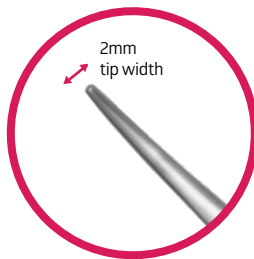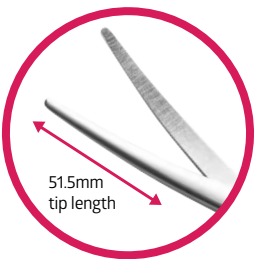

Metzenbaum Scissors Curved 160mm

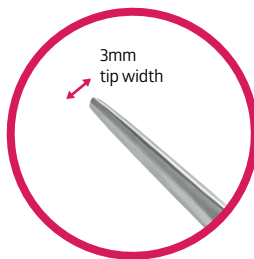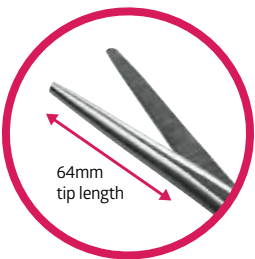

Metzenbaum Scissors Straight 200mm

NEW

SUPERCUT BABY METZENBAUM SCISSORS

SZSC2121	SuperCut Baby Metzenbaum Scissors Curved 120mm	20 units
----------	--	----------

- Ideal for fine cutting of soft tissue and blunt dissection
- One serrated blade and one sharp cutting blade allows a superior grip and cut

IMAGE COMING SOON

EXTENDED RANGE


Colposcopy LLETZ Ball and Loop Electrodes

Colposcopy LLETZ Ball Electrodes

- Single-use ensures no cross infection risks or health and safety standards
- Gap between plastic and ball electrode for better access
- 152mm Ball Electrodes suitable for larger patient
- Non-Stick Teflon coating changes the heat dissipation and prevents tissue from sticking to the instrument

Colposcopy LLETZ Loop Electrodes

- Tungsten loops give ideal tension for greater control and are stiffer at smaller dimensions which prevents the wire from folding when cutting
- High cutting performance with reduced thermal damage to tissue
- Recognised colour coding with loop size printed on the handle to easily distinguish between the variants
- Universal shaft fits most electrosurgical pencils
- Single-use ensures first time sharpness and performance with no cross infection risks or health and safety standards

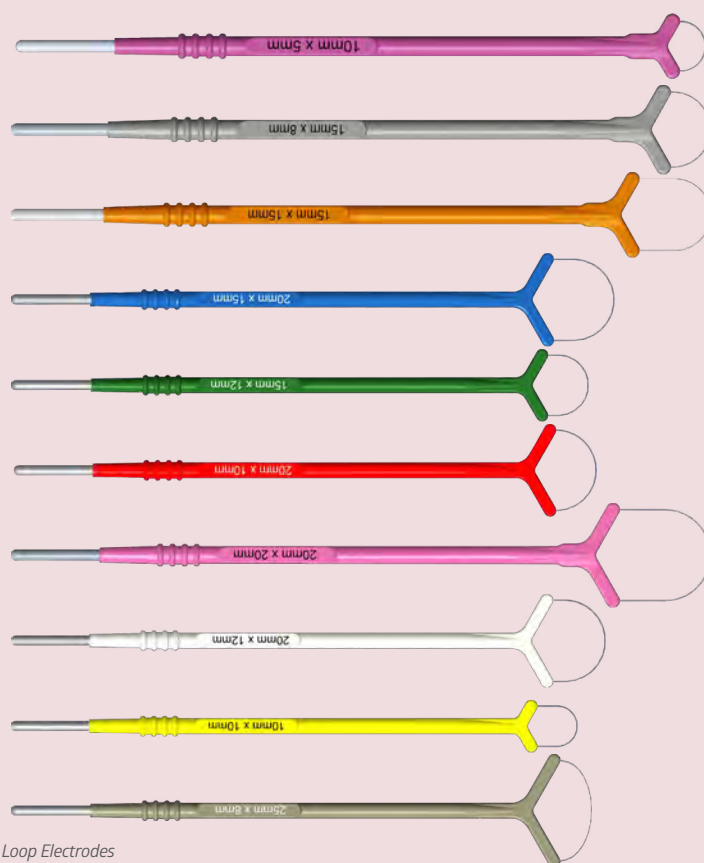

Loop Electrodes

Gap between plastic and ball electrode



Ball Electrodes

LLETZ BALL ELECTRODES

BET11303	Ball Electrode, 3mm diameter, Non-Stick Teflon Coated, 130mm length	10 units
BET11305	Ball Electrode, 5mm diameter, Non-Stick Teflon Coated, 130mm length	10 units
BET11503	Ball Electrode, 3mm diameter, Non-Stick Teflon Coated, 152mm length	10 units
BET11505	Ball Electrode, 5mm diameter, Non-Stick Teflon Coated, 152mm length	10 units

LLETZ LOOP ELECTRODES

LEC1130505-PU	Loop Electrode, 5x5mm, 130mm Shaft (Purple)	10 units
LEC1131005-LP	Loop Electrode, 10x5mm, 130mm Shaft (Light Purple)	10 units
LEC1131010-YE	Loop Electrode, 10x10mm, 130mm Shaft (Yellow)	10 units
LEC1131508-GY	Loop Electrode, 15x8mm, 130mm Shaft (Grey)	10 units
LEC1131512-GN	Loop Electrode, 15x12mm, 130mm Shaft (Green)	10 units
LEC1131515-OR	Loop Electrode, 15x15mm, 130mm Shaft (Orange)	10 units
LEC1132010-RE	Loop Electrode, 20x10mm, 130mm Shaft (Red)	10 units
LEC1132012-WH	Loop Electrode, 20x12mm, 130mm Shaft (White)	10 units
LEC1132015-LB	Loop Electrode, 20x15mm, 130mm Shaft (Light Blue)	10 units
LEC1132020-PI	Loop Electrode, 20x20mm, 130mm Shaft (Pink)	10 units
LEC1132508-TA	Loop Electrode, 25x8mm, 130mm Shaft (Tan)	10 units


Dental Syringes

SPECIAL SIZES

DENTAL SYRINGES

DGS2001	Dental Syringe 85mm for 2.2ml vial	20 units
DGSL2001	Dental Syringe Long 115mm for 2.2ml vial	20 units

- Suitable for use where needle stick injury is a concern (compliant with EU directive 2010/32/EU – prevention from sharps injuries)
- Popular top loading style to accommodate 2.2ml vials
- Barrel diameter is wide enough to take most vials
- Available in extra long length for use with larger, high BMI patients
- Designed for administering anaesthetic during a wide range of procedures

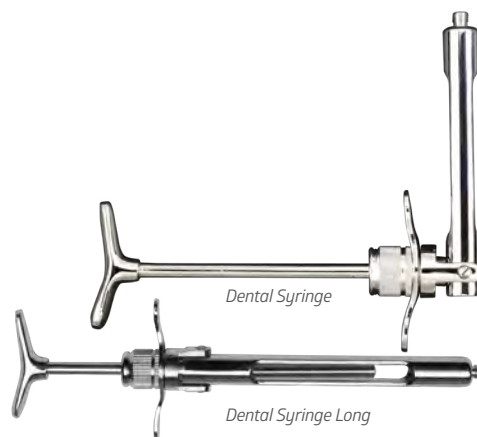


Endocervical Specula

KOGAN ENDOCERVICAL SPECULUM

ECF1001	Kogan Endocervical Specula with ratchet	10 units
---------	---	----------


- Specula with 275mm reach and 25mm jaws
- Removes the difficulty of cleaning hinge and ratchet
- Scaled ratchet for controlled visibility

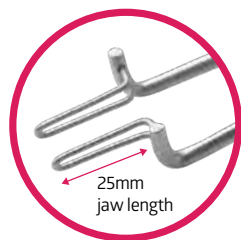

KURIHARA ENDOCERVICAL SPECULUM

KSB1001	Kurihara Endocervical Specula black	10 units
---------	-------------------------------------	----------


- Strong metal construction
- Specially engineered to enable a quick and accurate cervical dissection
- Ergonomic design that allows a secure grip when holding tissue

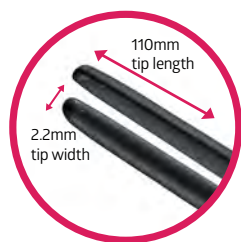

SIMS UTERINE SOUND

USH2001	Sims Uterine Sound	20 units
---------	--------------------	----------

- Malleable for ease of use
- 300mm shaft with 10mm graduations

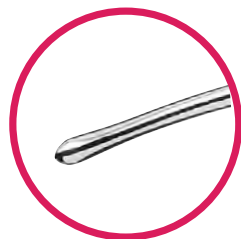

Metal Cusco Specula

- Extended range of sizes caters for all patient types with Small XXLong, Medium Long, Large Long, Large Extra Long, and Broad in addition to the standard sizes usually available
- Insulated & non-insulated options available
- Smooth robust design for improved patient and user safety
- Robust self-retaining screw that holds in place

CUSCO INSULATED SPECULA WITH SMOKE TUBE

CSI200S	Cusco Specula Insulated with Smoke Tube Small	20 units
CSI200M	Cusco Specula Insulated with Smoke Tube Medium	20 units
CSI200ML	Cusco Specula Insulated with Smoke Tube Medium Long	20 units
CSI200L	Cusco Specula Insulated with Smoke Tube Large	20 units
CSI100LL	Cusco Specula Insulated with Smoke Tube Large Long	10 units
CSI100LE	Cusco Specula Insulated with Smoke Tube Large Extra Long	10 units
CSI100B	Cusco Specula Insulated with Smoke Tube Broad	10 units

- Durable & improved insulation performance
- Enables hot Colposcopy procedures to be performed
- Strong metal construction with improved insulation performance for enhanced safety



Cusco Specula Insulated with Smoke Tube Medium

CUSCO NON-INSULATED SPECULA WITH SMOKE TUBE

CSM200S	Cusco Specula Metal with Smoke Tube Small	20 units
CSM200M	Cusco Specula Metal with Smoke Tube Medium	20 units
CSM200ML	Cusco Specula Metal with Smoke Tube Medium Long	20 units
CSM200L	Cusco Specula Metal with Smoke Tube Large	20 units
CSM100LL	Cusco Specula Metal with Smoke Tube Large Long	10 units
CSM100LE	Cusco Specula Metal with Smoke Tube Large Extra Long	10 units
CSM100B	Cusco Specula Metal with Smoke Tube Broad	10 units



Cusco Specula Metal with Smoke Tube Medium

Metal Cusco Specula

EXTENDED RANGE


CUSCO NON-INSULATED SPECULA WITHOUT SMOKE TUBE

CMM200S	Cusco Specula Metal Small	20 units
CMM200M	Cusco Specula Metal Medium	20 units
CMM200ML	Cusco Specula Metal Medium Long	20 units
CMM200L	Cusco Specula Metal Large	20 units
CMM100SE	Cusco Specula Metal Small XXLong	10 units
CMM100LL	Cusco Specula Metal Large Long	10 units
CMM100LE	Cusco Specula Metal Large Extra Long	10 units
CMM100B	Cusco Specula Metal Broad	10 units

- Small XXLong Metal Cusco Specula has been specially designed to improve the examination of post-menopausal women



Width x Length (mm)	Insulated Smoke Tube	Metal Smoke Tube	Metal w/out Smoke Tube
SMALL 25 X 85mm	CSI200S	CSM200S	CMM200S
MEDIUM 30 X 85mm	CSI200M	CSM200M	CMM200M
MEDIUM LONG 30 X 95mm	CSI200ML	CSM200ML	CMM200ML
LARGE 35 X 95mm	CSI200L	CSM200L	CMM200L
SMALL XXL 25 x 135mm	-	-	CMM100SE
LARGE LONG 35 X 110mm	CSI100LL	CSM100LL	CMM100LL
LARGE XL 35 X 115mm	CSI100LE	CSM100LE	CMM100LE
BROAD 40 X 95mm	CSI100B	CSM100B	CMM100B

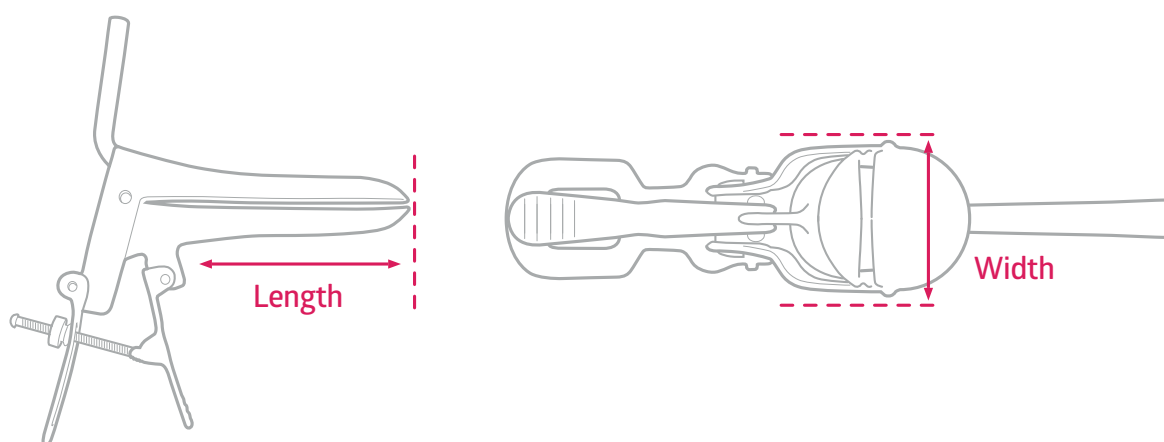


Electrosurgical Pencils

ELECTROSURGICAL PENCILS

EPF2573-H Foot Switch with Holster, 70mm Stainless Steel Blade Electrode, 3.0m cable 25 units

- Used in conjunction with Colposcopy Needles, LLETZ Ball and Loop Electrodes which attach to the Pencil
- Our Pencils provide an effective Current for Cauterisation

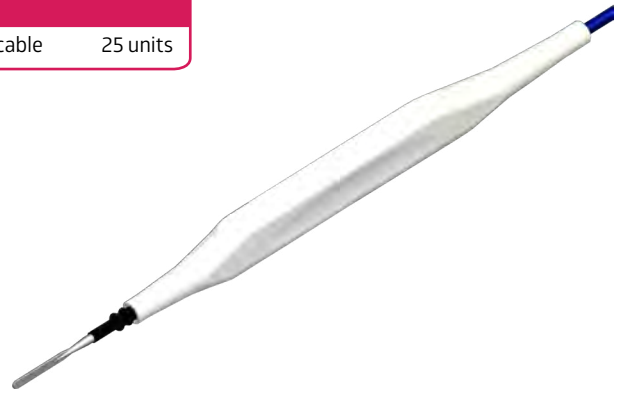

Scope Warmer

THEATRE EFFICIENCY

SCOPE WARMER

31SW10	ClearView™ Anti-Fog Scope Warmer, sterile packed for single-use	10 units
31SW30	ClearView™ Anti-Fog Scope Warmer, sterile packed for single-use	30 units

- Simple preheating to eliminate condensation
- Secures the surgeon's vision
- Warms full scope length to 37° within 90 seconds, staying above body temperature for 120 minutes
- Long-acting to avoid disruption or repeated defogging

Click disk to start warming



Instructional poster available
Contact us on +44 (0)1792 797910



Instrument Wipes

NETCELL® PVA INSTRUMENT WIPE

35-900 PVA Instrument Wipe 20 units

- Quickly removes protein, blood and particles without damage to the instrument
- Manufactured from ultra-smooth, biocompatible PVA sponge that will not snag
- Soft, strong, flexible, fibre and snag free material



“Supporting visibility, quality and safety in your supply chain”**

Our instruments' packaging features the latest machine readable GS1-standard barcoded labels for product traceability.

The newly incorporated labelling technology is a timely development with an increasing number of health providers worldwide adopting an 'eProcurement' strategy and will provide machine readable product information on all labels to meet expected EU and other regulators demands.

Associated benefits include:

- Improve patient care and reduce costs
- Improve stock control
- One global language
- Accurately track equipment

“Unique device identification (UDI) compliance represents a significant development for our customers. Automation of information flow will have influences across the whole purchasing and supply process, providing the benefits of superior instrument traceability and improved patient safety.”

“We are working closely with Kodit UDI software to provide our labelling solutions to meet the latest international regulations and we serve as a reference site for other manufacturers to learn from.”

DTR Medical

For more information on our new product labelling, please contact Customer Services on +44 (0) 1792 797910 or dtrinfo@innoviamedical.com

** - GS1 – Global Traceability Standards

Matching reusable standards

We benchmark against reusable British and international standards:

BS 5194-2:1989

Specification for Instruments with Pivot Joints

BS 5194-3:1985

Specification for Dissecting Forceps

BS 5194-4:1985

Specification for Scissors, Shears and other Jointed Instruments

We operate our own UK production facilities equipped with ISO class 7 cleanrooms

DTR Medical

Gynecology - Single-Use Instruments

DTR Medical Ltd

An Innovia Medical Company

17 Clarion Court, Enterprise Park, Swansea SA6 8RF UK

Telephone: +44 (0)1792797910

dtrinfo@innoviamedical.com

US 1.888.229.2875 | **International** +44 (0)1792797910

www.innoviamedical.com/single-use-instruments