
Safety Applicator
Applicateur de sécurité
Sicherheitsapplikator
Aplicador de seguridad

Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso

Figure 1 | Figure 1 | Abbildung 1 | Figura 1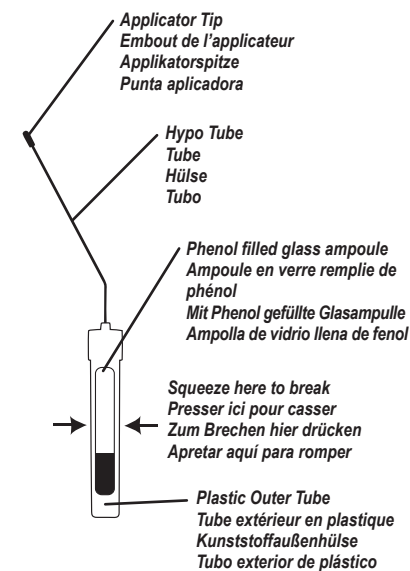

Hold upright and squeeze both sides.
 Tenir à la verticale et presser des deux côtés.
 Aufrecht halten und beide Seiten zusammendrücken.
 Sostener en posición vertical y apretar ambos lados.

Before using, read the following information:

NON-STERILE

Description:

The PhenolPro Safety Applicator is a single use applicator used to apply phenol. The liquefied phenol is a caustic liquid containing <11% water and >89% phenol.

For Professional Use Only.

How Supplied:

Liquefied phenol is supplied in a single use applicator containing 0.27 ml of liquefied phenol. Due to the small amount of phenol in tube, it may take time to dispense.

Storage:

Room Temperature. Protect from light and heat.

Instructions for Use:

The ampoule is closed and phenol is not in contact with the applicator.

1. Wear gloves when handling.
2. Open the pouch and remove the applicator.
3. Hold upright (applicator tip up, ampoule down) and squeeze the plastic tube by applying pressure to the middle of the ampoule inside the tube. (See Fig. 1) This breaks the ampoule releasing phenol into the plastic tube.
4. Turn the applicator over (applicator tip down, ampoule up) to start the phenol flowing towards the applicator tip.
5. The phenol is delivered into the hypo tube and onto the applicator tip by squeezing the plastic tube (See Fig. 1)
6. Place the applicator tip near the desired application site prior to squeezing the plastic tube.
7. The applicator is ready to use when the applicator tip is wet.
8. After use, place applicator back into foil pouch and dispose of in medical waste.

Warnings:

Local inflammation may occur in individuals with hypersensitivity to phenol. Liquefied phenol is absorbed directly through the skin and prolonged contact may cause chemical burns. Contact over large areas may cause systemic toxicity. (See Precautions)

Precautions:

Contact of liquefied phenol with the skin should be avoided as it may cause severe irritation with second to third degree burns; areas usually turn white and later yellowish-brown and may be deeply eroded and scarred. Solutions are readily absorbed through the skin and may cause profuse perspiration, intense thirst, nausea and vomiting, diarrhea, cyanosis from methemoglobinemia, hyperactivity, stupor, hypotension, hyperpnea, abdominal pain, hemolysis, convulsions, coma, and pulmonary edema followed by pneumonia. If death from respiratory failure is not immediate, jaundice and oliguria or anuria may occur. Skin sensitization occurs occasionally. A profound fulminating central nervous system depression with coma, hypothermia, loss of vasoconstrictor tone, cardiac depression, cerebral edema and respiratory arrest are common manifestations of systemic poisoning. However, stridorous breathing, mucus rales, froth at the mouth and nose eventually occur. Liver, kidney and bladder damage occur.

Adverse Reactions:

Phenol causes blanching and corrosion.

Clinical Pharmacology:

Liquefied phenol is both an anesthetic and an antiseptic. Liquefied phenol is also bactericidal and fungicidal with some sporicidal and virucidal activity.

Phenol is absorbed through the gastrointestinal tract and directly through the skin and mucous membranes. It is metabolized to phenyl-glucuronide and phenyl sulfate and small amounts are oxidized to catechol and quinol which are mainly conjugated. The metabolites are excreted in the urine and when oxidized to quinines may tint the urine green.

Overdosage:

Topically applied liquefied phenol can be absorbed in sufficient amounts to provide systemic effects.

(See Precautions)

For treatment of skin contact remove contaminated clothing and shoes immediately. Wash effected area with soap or mild detergent in large amounts of water until no evidence of chemical remains (at least 15-20 minutes). In case of chemical burns, cover areas with sterile, dry dressings. Bandage securely but not too tightly and seek medical attention immediately. For eye contact wash eyes immediately with large amounts of water, occasionally lifting upper and lower lids until no evidence of chemical remains (at least 15 – 20 minutes). In case of burns apply sterile bandages loosely without medication and see medical attention immediately.

In case of ingestion, if the victim is conscious, and if corrosive injury is absent, remove poison by gastric lavage or emesis. Activated charcoal is useful. Follow with 60ml of castor oil which dissolves phenol, retards its absorption and hastens its removal. Follow castor oil by giving 30 – 60 ml of Fleet's phosphor-soda diluted 1:4 in water. Gastric lavage and emesis are not to be used in the presence of esophageal injury. Gastric lavage or emesis should not be performed on an unconscious person. Gastric lavage should be performed by qualified medical personnel. Medical attention should be sought immediately.

No specific antidote is available. Treat symptomatically and supportively.

Patent pending.

Single-use, one-patient device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of the patient.

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