SAFETY APPLICATOR
Instructions for Use

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 **BEFORE USING, READ THE FOLLOWING INFORMATION**

 **EN**

NON-STERILE

DESCRIPTION

The PhenolPro™ Safety Applicator consists of a stainless-steel hypodermic tube and foam applicator tip. A glass ampoule filled with liquefied phenol, encased in a plastic outer tube is inserted into the hypodermic tube. The liquefied phenol is a caustic solution containing <11% water and >89% phenol.

FOR PROFESSIONAL USE ONLY

STORAGE

Room Temperature. Protect from light and heat.

INTENDED USE/INDICATIONS FOR USE

The device is indicated to provide a topical anesthetic to the tympanic membrane.

WARNINGS/PRECAUTIONS

- Single use device. Do not re-use.
- Wear gloves when handling
- Local inflammation may occur in individuals with hypersensitivity to phenol.
- Care should be taken when using liquefied phenol as it may cause severe irritation to the skin. Contact over large areas may cause systemic toxicity.

INSTRUCTIONS FOR USE

CAUTION: Wear gloves when handling

1. Ensure that the ampoule is closed, and the liquefied phenol is not in contact with the applicator
2. Open the pouch and remove the applicator
3. Hold upright (applicator tip up, ampoule down) and squeeze the plastic tube by applying pressure to the middle of the ampoule inside the tube
4. Turn the applicator over (applicator tip down, ampoule up) to start the liquefied phenol flowing towards the applicator tip.
5. Once the applicator is turned over, the liquefied phenol will flow into the hypodermic tube and onto the applicator tip by itself after a few seconds without squeezing. If after several seconds the liquefied phenol does not flow to the applicator tip, gently squeeze the housing to encourage the liquefied phenol to flow. The applicator is ready to use when the applicator tip is wet.

CAUTION: Do not squeeze the housing repeatedly or with too much force as glass shards may angle through the plastic housing if too much pressure is applied

6. Insert the applicator into the ear canal and apply liquefied phenol to the tympanic membrane
7. Carefully remove the applicator and dispose per instructions.

DISPOSAL

After use, place the applicator back in the foil pouch and dispose of in accordance with the facility's procedure. The device comes into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination.

INCIDENT REPORTING

Any incident that has occurred in relation to the device shall be reported to the manufacturer and appropriate medical device regulatory authority in which the user and/or patient is established.

CLINICAL PHARMACOLOGY:

Liquefied phenol is both an anesthetic and an antiseptic. Liquefied phenol is also bactericidal and fungicidal with some sporicidal and virucidal activity.

Phenol is absorbed through the gastrointestinal tract and directly through the skin and mucous membranes. It is metabolized to phenyl-glucuronide and phenyl sulfate and small amounts are oxidized to catechol and quinol which are mainly conjugated. The metabolites are excreted in the urine and when oxidized to quinines may tint the urine green.

OVERDOSAGE

Typically applied liquefied phenol can be absorbed in sufficient amounts to provide systemic effects. The solution is readily absorbed through the skin and may cause profuse perspiration, intense thirst, nausea and vomiting, diarrhea, cyanosis from methemoglobinemia, hyperactivity, stupor, hypotension, hyperpnea, abdominal pain, hemolysis, convulsions, coma, and pulmonary edema followed by pneumonia. If death from respiratory failure is not immediate, jaundice and oliguria or anuria may occur. Skin sensitization occurs occasionally. A profound fulminating central nervous system depression with coma, hypothermia, loss of vasoconstrictor tone, cardiac depression, cerebral edema and respiratory arrest are common manifestations of systemic poisoning. However, stridorous breathing, mucus rales, froth at the mouth and nose eventually occur. Liver, kidney and bladder damage occur.

For treatment of skin contact remove contaminated clothing and shoes immediately. Wash effected area with soap or mild detergent in large amounts of water until no evidence of chemical remains (at least 15-20 minutes). In case of chemical burns, cover areas with sterile, dry dressings. Bandage securely but not too tightly and seek medical attention immediately. For eye contact wash eyes immediately with large amounts of water, occasionally lifting upper and lower lids until no evidence of chemical remains (at least 15 - 20 minutes). In case of burns apply sterile bandages loosely without medication and see medical attention immediately.

In case of ingestion, if the victim is conscious, and if corrosive injury is absent, remove poison by gastric lavage or emesis. Activated charcoal is useful. Follow with 60ml of castor oil which dissolves phenol, retards its absorption and hastens its removal. Follow castor oil by giving 30 - 60 ml of Fleet's phosphor-soda diluted 1:4 in water. Gastric lavage and emesis are not to be used in the presence of esophageal injury. Gastric lavage or emesis should not be performed on an unconscious person. Gastric lavage should be performed by qualified medical personnel. Medical attention should be sought immediately.

No specific antidote is available. Treat symptomatically and supportively.

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