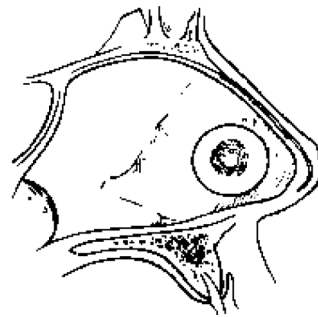
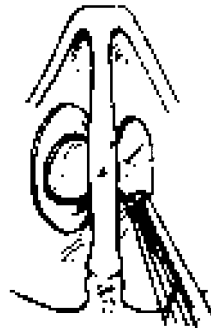
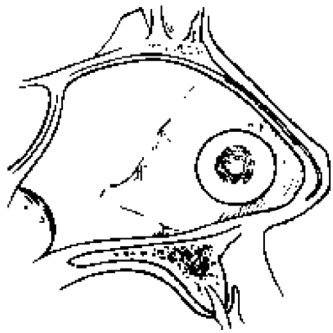
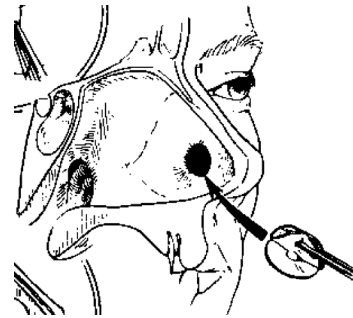
Nasal Septal Button
Bouton pour septum nasal
Nasenseptumknopf
Tapón para tabique nasal
Otturatore per setto nasale
Nasale septumknop
Neseseptumknapp

Instructions for Use
 Mode d'emploi
 Gebrauchsanweisung
 Instrucciones de uso
 Istruzioni per l'uso
 Gebruiksaanwijzing
 Bruksanvisning

Summit Medical
Nasal Septal Button

(EN)

 Before using, read the following information:

DESCRIPTION

The Nasal Septal Button is a silicone elastomeric device and is used for non-surgical closure of septal perforations.

INDICATIONS FOR USE

Where temporary, non-surgical closure of a nasal septal perforation is desired.

CONTRAINDICATIONS

None known. This device is not designed, sold or intended for use except as indicated or prescribed by a physician.

POSSIBLE ADVERSE REACTIONS

Localized infection.

PRECAUTIONS

The Nasal Septal Button intended for single use only and should not be resterilized. Do not use if pouch is damaged.

INSTRUCTIONS FOR USE

1. Trim to size. Avoid forming sharp corners.
2. Place button through septal perforation.
3. Ensure that flanges are positioned comfortably on either side of the septal wall.
4. The septal button may be removed and cleaned as necessary.

CLEANING

The septal button may be removed and cleaned as necessary using non-oily soap and rinsed thoroughly.

IUSA Rx Only
 For US Audiences Only

Single-use, one-patient device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of the patient.

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