Focal·One®

ROBOTIC FOCAL HIFU

- Ⓔ EN FOCALONE HIFU PROBE TRANSPORT TRAY **INSTRUCTIONS FOR USE: English**
- Ⓕ FR PLATEAU DE TRANSPORT DE SONDE FOCALONE HIFU **MODE D'EMPLOI: Français**
- Ⓖ DE FOCALONE HIFU-SONDEN-TRANSPORTSCHALE **GEBRAUCHSANWEISUNG: Deutsch**
- Ⓘ IT VASSOIO PER IL TRASPORTO PER SONDA FOCALONE HIFU **ISTRUZIONI PER L'USO: Italiano**
- Ⓔ ES BANDEJA DE TRANSPORTE DE SONDA FOCALONE HIFU **MODO DE EMPLEO: Español**

For an electronic version of the IFU, visit:

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INTENDED USE

The FocalOne® HIFU Probe Transport Tray is intended to contain and protect the FocalOne® HIFU Probes TMS 902081 and TMS 253728 during transport from the operating room to the sterile processing department.

It is designed to hold one FocalOne® HIFU Probe and is compatible with automated washers for repeated cleaning cycles.

These products are not intended to serve as a sterile barrier or for sterilization while assembled.

INDICATIONS FOR USE

The FocalOne® HIFU Probe Transport Tray is indicated for use when the FocalOne® HIFU Probe requires secure containment and protection during transport from the operating room to the sterile processing department. The transport tray can be processed in an automated washer; however, the probe **MUST** be removed from the tray prior to washing.

INTENDED USER

The transport tray is intended to be used by healthcare professionals in the operating room and sterile processing departments as an accessory for transportation of the FocalOne® HIFU Probe.

MATERIAL

The transport tray is reusable and made of a durable, formed polycarbonate plastic.

DESCRIPTION

The transport tray consists of a base formed to the FocalOne HIFU Probe and a removable cover.

TRANSPORT TRAY USE

1. Clean and inspect prior to use
2. Set the transport tray on flat surface.
3. Place FocalOne® HIFU Probe in the transport tray by laying the probe (with protective cover over distal end) into appropriate cavity. Coil the cords and lay into the cord basin. Lay the connectors into their designated cavities.
4. Place lid on the transport tray and ensure the lid snaps on.
5. Transport the tray and its contents.
6. Open lid.
7. Remove FocalOne® HIFU Probe.
8. Process per the probe's instruments/materials IFU.

WARNINGS AND PRECAUTIONS

1. **DO NOT** soak the probe
2. **DO NOT** autoclave the probe or transport tray.
3. **DO NOT** drop, as this may cause damage to the probe inside and/or the transport tray.
4. **DO NOT** use if package is damaged or unintentionally opened prior to use.
5. **DO NOT** use as a sterile barrier.
6. Clean and inspect prior to use. Transport tray may be reused until mechanical failure occurs.

Life Cycle: DO NOT use if transport tray is cracked or broken.

Disclaimer: Transport tray is warranted against defects in materials or workmanship for 1 year.

7. **DO NOT** wash transport tray with a hard brush. This can cause damage to the transport tray

MANUAL CLEANING

1. Use a neutral (6.0 – 8.5) pH detergent
2. Remove all visible soil and contaminants using a soft brush
3. Rinse thoroughly with clean water to remove detergent
4. Dry completely before storage

AUTOMATED WASHER CLEANING

1. Transport tray is safe for use in automated washers and can withstand up to 190°F (88°C)
2. The FocalOne® HIFU Probe **MUST** be removed from the transport tray prior to using an automated washer.

STORAGE AND HANDLING

Store the FocalOne® HIFU Probe Transport Tray in a clean, dry environment at ambient temperature after drying completely. Protect from physical damage during storage and transport.

DISPOSAL

In the event the transport tray does not pass inspection prior to use or has otherwise been deemed no longer fit for purpose, the device shall be disposed of in line with local protocol. The method of disposal shall depend on the potential risks of cross-contamination and infection when the need for disposal is identified.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the FDA/competent authority of the Member State in which the user and/or patient is established.

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MDSS GmbH
Summit Medical LLC